


Instructions for Adsoft™ Plus Eartip Conversion

If you have an ADSCOPE™ manufactured prior to 2014 (or any ADSCOPE™ other than the ADSCOPE™ Sprague) with threaded aural tubes, you will need to affix a special adapter to use the new ADSOFT™ Plus fitted silicone eartips. Please follow these 3 simple connection steps for affixing your new eartips.

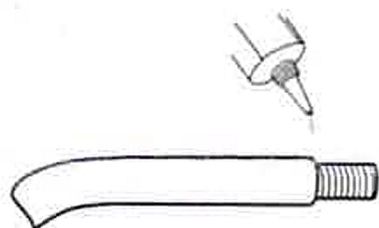

Step 1.


Place a single drop of glue*
on the threads of the binaural.

*(Loctite®, Krazy® Glue, Clear Nail Polish)



Step 2.

Thread on earpiece adapter,
until hand tight.

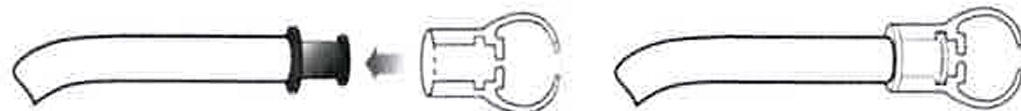

Step 3.

Slide Adsoft™ eartip over adapter until it locks tight
into internal channel. Repeat for second binaural.