

READY HEAT

INFANT WARMING COCOON

UP TO 8 HOURS OF HEAT
WARMS TO 100°F (38°C)



INSTRUCTIONS FOR USE

1. DO NOT open the outer protective package until ready to use.
2. Remove the cocoon from the outer protective package and expose the Ready-Heat Infant Cocoon to air. DO NOT open cocoon pockets or inside pouch.
3. Cocoon will start to heat immediately. Allow 10-15 minutes to reach operating temperature.
4. If heat decreases, expose cocoon to air.
5. After use, dispose of cocoon with regular waste. All natural ingredients; will not harm environment.



WARNINGS **Failure to follow these precautions may result in serious burns or other serious injuries.**

1. **DO NOT use cocoon on exposed skin.**
2. Careful supervision is required for use with the elderly, infants, children, people with sensitive skin and those who can't feel heat or regulate heat. Keep out of reach of children.
3. People with allergies, diabetes, frostbite, scars, open wounds or circulatory problems should consult a physician before using the Ready-Heat Infant Cocoon.
4. Remove immediately if cocoon becomes too hot or if the infant is uncomfortable. Maximum temperature may reach 115°F (46°C). DO NOT use while sleeping.
5. DO NOT open the sealed pockets or pouch and DO NOT allow the contents to contact mouth or eyes. If contact with skin or eyes, wash with clean water. Contents contain iron powder which, if swallowed, can be harmful. If contents are swallowed, immediately call a poison control center.
6. DO NOT fold cocoon or place objects on top of heating elements as folding or pressure may increase heat.
7. DO NOT use in oxygen rich environment, including hyperbaric chamber or oxygen tent.

CONTENTS: Iron powder, water, activated carbon, wood powder, salt and polyethylene foam

We recommend the Ready-Heat Infant Warming Cocoon be used in conjunction with our Ready-Heat Infant Warming Mattress, the Mattress to be placed outside & underneath for constant warming should the Cocoon need to be opened for patient access.



TechTrade

Life-Assist
Helping Heroes Save Lives



TechTrade, LLC ▪ 6900 Tavistock Lakes Blvd. ▪ Suite 400 ▪ Orlando, FL 32827 ▪ USA
1-866-491-TECH ▪ Fax: 201-706-8131 ▪ info@techtradellc.com ▪ www.ready-heat.com

SIZE: 31.5" x 28" (80 cm x 71 cm)
PART# GIW6C
WEIGHT: 12 oz. (340 gm)



EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands



(01)00850017905085

Please visit www.ready-heat.com/IFU
for complete IFU and Symbol Descriptions.

MADE IN USA

0459

CLEARED

RHPLG-6 rev 6, June 2021