

Pediatric Intubation Trainer

Directions for Use



Laerdal[™]
helping save lives

Table of Contents

Items Included	4
Skills Taught	4
Laerdal Recommends	5
Instructions for Use	5
Care and Maintenance	6
Replacement Parts	6
Global Warranty	6

The Pediatric Intubation Trainer (255-00001) realistically simulates the head and upper body of a six-year-old hospital patient. It is specifically designed for training professionals in the field of pediatric airway management techniques.

The manufacturing excellence and ease of replacing individual parts of this trainer should provide years of training when reasonable care and maintenance is practiced.

Items included:

- (1) Pediatric Intubation Trainer
- (1) Manikin Lubricant
- (1) Carry Case
- (1) Laerdal Global Warranty Booklet
- (1) Directions for Use

Skills taught:

- Oral intubation
- Nasal intubation
- Digital intubation
- Bag/Valve Mask
- Oropharyngeal airway insertion
- Naso pharyngeal airway insertion

Laerdal Recommends

Endotracheal tube size 4 or 5, cuffed or uncuffed

Manikin lubricant or liquid soap

Laryngoscope blade, size 2 straight or curved blade

Laryngoscope handle

Instructions for Use:

For intubation of the pediatric intubation head, the following equipment is recommended.

- a. Endotracheal tube size 4 or 5, cuffed or uncuffed
- b. Manikin lubricant or liquid soap
- c. Laryngoscope blade, size 2 straight or curved blade
- d. Laryngoscope handle

Laerdal Recommends

Prior to airway placement, all tubes and passages must be lubricated with a spray of Manikin Lubricant or mild liquid soap.

Care and Maintenance:

1. Clean with mild soap and water
2. Use only on clean surface. Avoid felt tipped markers, ink pens, acetone, iodine or other staining products and avoid placing the trainer on newsprint or inked lines of any kind.
3. To ensure longevity, each trainer should be cleaned after each training session and a general inspection should be conducted regularly.
4. Store properly between teaching sessions.

Replacement Parts

For latest version of Spare Parts and Accessories, visit www.laerdal.com

Global Warranty

See the Laerdal Global Warranty Booklet, or www.laerdal.com



1001635

©Laerdal Medical, Gatesville TX. All rights reserved
Printed in USA
1001635 rev E



Laerdal
helping save lives