INFORMACIÓN REGLAMENTARIA

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Distributed in the USA by:
LAERDAL MEDICAL CORPORATION
Wappingers Falls, New York 12590-8840
Tel. (800) 431-1055, (845) 297-7770
Fax (800) 227-1143, (845) 298-4545
E-mail: customerservice@laerdal.com

Distributed in Canada by:
LAERDAL MEDICAL CANADA LTD.
151 Nashdene Rd., Unit #45
Toronto, ON, Canada, M1V 4C3
Tel. (416) 298-9600
E-mail: savelives@laerdal.ca

Patentado – Patente U.S. n° 6.170.486
REF Cat. n° 700-00001
CANT. 30 unidades

LOT

REGULATORY INFORMATION

 Single Use Warning: Designed for single use only. Do not re-use. Re-use will lead to increased risk of cross contamination, degradation of performance and/or device malfunction. Laerdal is not responsible for any consequences of re-use.

Non Sterile.
Latex free
Sta-Blok is a trademark of Laerdal Medical AS
Product: Sta-Blok Head Immobilizer with Standard and Severe Weather Straps
Made in Mexico

MANUFACTURED FOR LAERDAL MEDICAL AS
P.O. Box 377, Tanke Svilandsgate 30, 4002 Stavanger, Norway

Distributed in the USA by:
LAERDAL MEDICAL CORPORATION
Wappingers Falls, New York 12590-8840
Tel. (800) 431-1055, (845) 297-7770
Fax (800) 227-1143, (845) 298-4545
E-mail: customerservice@laerdal.com

Distributed in Canada by:
LAERDAL MEDICAL CANADA LTD.
151 Nashdene Rd., Unit #45
Toronto, ON, Canada, M1V 4C3
Tel. (416) 298-9600
E-mail: savelives@laerdal.ca

Patented – U.S. Patent No. 6,170,486
REF Cat. No. 700-00001
QTY 30 pieces

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STA-BLOK

HEAD IMMOBILIZER**DIRECTIONS FOR USE**

The Sta-Blok Head Immobilizer is to be used as a part of a total immobilization system in conjunction with other immobilization products and techniques to help reduce potential for movement of the patient's head and neck in cases where spinal fracture, dislocation and/or spinal cord injury is suspected.

 **WARNING:**

- In cases of suspected spinal cord injury, proper head immobilization is only one part of a total spinal immobilization process.
- Only qualified healthcare providers trained in the use of this and other immobilization products should use the Sta-Blok Head Immobilizer. Failure to use this product as directed, and under prescribed protocols from the local cognizant medical control authority, may result in patient injury and/or disability.

 **CAUTION:**

- All healthcare provider personnel should read and fully understand these instructions prior to using the product.
 - Store product in a cool, dry place.
 - Do not store in excessive heat or in direct sunlight.
-  *NOTE: These Directions are general guidelines and instructions for proper use of the Sta-Blok Head Immobilizer. They are not intended to be and should not be construed as teachings, recommendations or directions for clinical practice.*

COMPONENTS: EACH STA-BLOK PACKAGE CONTAINS THE FOLLOWING

- Occipital Pad - Foam pad with blue loop (hook and loop) fastener top surface.
- Side Support Blocks - Triangular foam blocks with hook (hook and loop) fastener surfaces.
- Standard Strap - White loop (hook and loop) fastener strap.
- Severe Weather Strap - Blue self-adhesive (peel and stick) fastening tape.
- Directions For Use leaflet.

POSITIONING THE STA-BLOK OCCIPITAL PAD

- Place the spineboard on a stable surface. Pull the liner from the Occipital Pad. Press Pad down firmly to secure it to the Spineboard.
- Place the patient on the spineboard. Use an extrication collar if available.

POSITIONING SIDE SUPPORT BLOCKS

- Place a Side Support Block against either side of the head, just above the ears.
- Press the Blocks firmly down onto the Occipital Pad.

SECURING PATIENT'S HEAD

- Apply the Standard Strap with the foam centered on patient's forehead.
- Attach each hook and loop end to the Side Support Blocks.

FOR SEVERE WEATHER

Remove the release liner from the Severe Weather Strap. Ensure that the strap is placed over the Standard Strap to avoid skin contact. Wrap the ends of the Severe Weather Strap around the spineboard and attach it to the back of the spineboard.

ENSURING A NEUTRAL POSITION

For children under 8 years old or a patient with helmet, use spinal alignment padding if needed to obtain a neutral position for the spine.

FOR EXTRA SUPPORT

The Severe Weather Strap may be used with the Standard Strap around the patient's chin or neck. The Severe Weather Strap is longer and therefore ideal for use on patients wearing motorcycle or football helmets.

 *NOTE: If circumstances prevent either the Standard and/or Severe Weather Straps from being used, suitable lengths of conventional medical adhesive or other tape may be substituted.*

TEMPERATURE

Operating temperature: 5°F to 102°F / -15°C to 38°C
Storage temperature: 32°F to 125°F / 0°C to 52°C
Recommend storage at room temperature