Little Sucker® Nasal Tip Nasal Suction Device

REF Catalog No.

N224 Premie
N224TP Premie
N225 Standard
N225TP Standard

DIRECTIONS FOR USE
INSTRUCTIONS D'UTILISATION
NÁVOD K POUŽITÍ
GEBRAUCHSANWEISUNG
ISTRUZIONI PER L'USO
INSTRUCCIONES DE USO
ZALECENIA DOTYCZĄCE
UŻYTKOWANIA

Manufactured by
NEOTECH PRODUCTS LLC
28430 Witherspoon Pkwy, Valencia, CA 91355, USA
1-800-966-0500 • neotechproducts.com

EMERGO EUROPE
Prinsessegracht 20
2514 AP, The Hague, The Netherlands

Made in USA

D453 Rev G

English

Little Sucker® Nasal Tip
Nasal Suction Device

FEATURES & BENEFITS

- Specialized tip designed to minimize potential tissue trauma
- Designed for single handed suctioning
- Thumb port for intermittent suctioning
- Soft, flexible tip similar to a bulb syringe
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

INDICATIONS FOR USE

The Little Sucker Nasal Tip is indicated for use as a vacuum powered suction device to aspirate fluids from the nares.

DIRECTIONS FOR USE

Step 1 Connect barbed end of Little Sucker Nasal Tip to suction line.

Step 2 Set suction to appropriate pressure per hospital protocol or physician recommendation.

Step 3 Block thumb port with thumb to begin suctioning.

Step 4 Suction nares with intermittent suctioning.

Note: For intermittent suctioning, unblock thumb port to stop suctioning. Block thumb port again to continue suctioning.

Step 5 Suction sterile water through Little Sucker Nasal Tip after each use.

Step 6 Replace Little Sucker Nasal Tip every 24 hours or per hospital protocol, whichever is sooner.

CAUTIONS & WARNINGS

Federal law restricts this device to sale by or on the order of a physician.

For nasal use only.

Inserting the Little Sucker Nasal Tip too far into the nares can cause tissue trauma.

The Little Sucker Nasal Tip should remain free of debris.

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