Neotech Meconium Aspirator Meconium Removal Device

REF Catalog No.

N0101
N0102

DIRECTIONS FOR USE
INSTRUCTIONS D'UTILISATION
NÁVOD K POUŽITÍ
GEBRAUCHSANWEISUNG
ISTRUZIONI PER L'USO
INSTRUCCIONES DE USO
ZALECENIA DOTYCZĄCE
UŻYTKOWANIA

 Manufactured by
NEOTECH PRODUCTS LLC
28430 Witherspoon Parkway, Valencia, CA 91355, USA
1-800-966-0500 • neotechproducts.com

EC REP EMERGO EUROPE
Prinsessegracht 20
2514 AP, The Hague, The Netherlands

Made in USA

D463 Rev D

English
Neotech Meconium Aspirator
Meconium Removal Device

FEATURES & BENEFITS

- Designed to eliminate mouth suctioning of meconium
- Suction control port for intermittent suctioning
- Attaches to most endotracheal tubes
- Featured in the Textbook of Neonatal Resuscitation¹
- N0102 (yellow) allows for easier identification
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

1) Textbook of Neonatal Resuscitation, 6th Edition (2011) pg. 43, Figure 2.7

INDICATIONS FOR USE

The Neotech Meconium Aspirator is intended for use as a suction device to aspirate meconium from a newborn's upper airway.

DIRECTIONS FOR USE

Step 1 Connect barbed end of Neotech Meconium Aspirator to suction line.

Step 2 Set the suction pressure at 80mm Hg or less.

Step 3 Intubate patient with proper ET tube.

Step 4 Connect 15mm I.D. end of Neotech Meconium Aspirator to 15mm O.D. ET tube adapter.

Step 5 Block thumb port with thumb to begin suctioning.

Step 6 Suction for two seconds or less at a time until meconium is full removed. Suction meconium intermittently.

Note: For intermittent suctioning, unblock thumb port to stop suctioning. Block thumb port again to continue suctioning.

Step 7 Continue to suction while ET tube is withdrawn.

Step 8 Discard after use.

CAUTIONS & WARNINGS

Federal law restricts this device to sale by or on the order of a physician.

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