

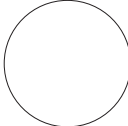





NASO-FLO[®] Sizing Chart



Below are some recommendations on how to properly size the Naso-Flo:

- » Before removing the tube from the bag, check the tip of the NPA. Verify the diameter of the tube is not larger than the nare of the patient.
- » Line up the tip of the Naso-Flo with the bottom of the patients ear lobe, and the top of the Naso-Flo with the bottom of their nose.
- » Always follow your local protocols.



	Size 4	ID 4.0 18 FR
	Size 5	ID 5.0 22 FR
	Size 6	ID 6.0 25 FR
	Size 7	ID 7.0 29 FR
	Size 8	ID 8.0 32 FR
	Size 9	ID 9.0 36 FR