BE PREPARED. PRESERVE LIFE.



GET TRAINED TO PRESERVE LIFE sammedical.com



BLEEDING GONTROL KII





SAM XT - INSTRUCTIONS FOR USE



Locate the site of injury and place the **SAM® XT** around the limb, directly onto the skin 2-3 inches above the wound.






in the buckle. Grip the strap close to the buckle and pull hard and steady against the buckle until it stops. Maintaining tension, press the strap back upon itself to hold it in place.

Insert the tip of the strap through the opening

DO NOT COVER WINDLASS C-HOOKS.

TWIST the metal windlass rod UNTIL THE BLEEDING HAS STOPPED.



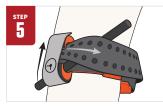

Fix the metal windlass rod inside the Windlass C-hooks and confirm that bleeding has stopped. If bleeding continues, try to further tighten the tourniquet with additional twists of the metal windlass rod. If this fails to stop the bleeding, apply a second **SAM® XT** side by side next to the first and ensure bleeding has stopped.

Wrap strap between Windlass C-hooks, over rod and around the limb.

Secure the (④) Time Band over Windlass C-hooks and record time of application.

TREATMENT BY WOUND LOCATION

