

TRAUMAGEL®

HEMOSTASIS REVOLUTIONIZED

Fast, Easy, and Effective Hemostasis
for Traumatic Bleeding




Addressing **Your Challenges:**

Speed of Hemostasis: TRAUMAGEL® controls bleeding in seconds, minimizing blood loss.

Ease of Use: Simple and intuitive deployment by any trained personnel.

Safety: Reduces the risk of injury from wound packing, enhancing safety for the operator.

Efficacy in Carotid Laceration (bovine)*

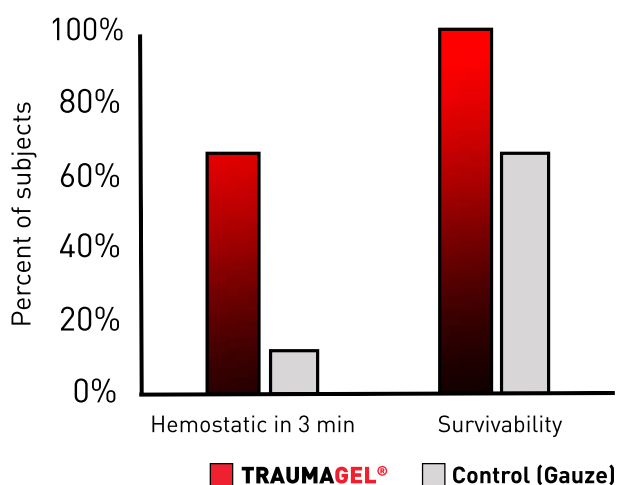

*Data on file at Cresilon

Testimonials

"I found the product revolutionary. I really enjoyed its ease of deploy ability and the efficacy was astounding."

**Emergency Medicine Physician,
EMS Medical Director**


"When it came time to remove the gel, as you would for definitive care in a hospital, I was surprised at how easily it came off."

**Emergency Medicine Physician,
Trauma Care**



TRAUMAGEL® IS FDA CLEARED FOR USE BY LICENSED MEDICAL PROFESSIONALS IN THE US ONLY. FOR MORE INFORMATION, CONSULT INSTRUCTIONS FOR USE.



Directions For Use

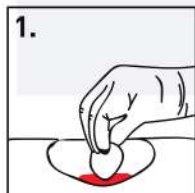



1. Identify the source of bleeding. Clear as much blood as possible from the wound site prior to application.

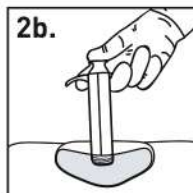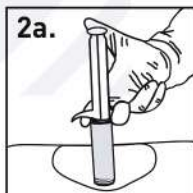

2. Tear open pouch and unscrew cap. Insert syringe deep into wound as close to bleeding source as possible. Firmly and quickly expel all contents into the wound. Keep empty package.



3. Pack gauze into the wound opening atop the applied gel.

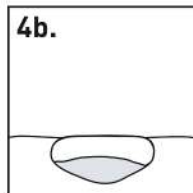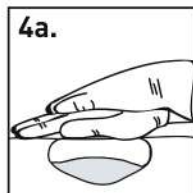

4. Apply **moderate palm compression**, covering as much surface area as possible. Continue **compression for three (3) minutes** or until hemostasis is achieved. Avoid ejecting any of the underlying gel. Apply additional gauze if needed. Show empty package to medical personnel.

To remove TRAUMAGEL®:

A. TRAUMAGEL® must be removed within 24 hours of application.

B. Remove all the gel using gauze and/or thorough irrigation.

Contraindications: TRAUMAGEL® is not intended for surgical use. TRAUMAGEL® is not intended to be used as a wound-closure device.

Refer to ifu.traumagel.com for full Instructions for Use

Warnings:

1. Do not apply TRAUMAGEL® on junctional wounds not amenable to pressure.

2. Do not apply TRAUMAGEL® unless the source of bleeding is easily visible. Do not blindly apply TRAUMAGEL®.

3. Do not use TRAUMAGEL® on penetrating injuries of the chest or abdomen.

TRAUMAGEL® IS FDA CLEARED FOR USE BY LICENSED MEDICAL PROFESSIONALS IN THE US ONLY. FOR MORE INFORMATION, CONSULT INSTRUCTIONS FOR USE.