

Ventlab LLC  
October 11, 2023

# **URGENT: MEDICAL DEVICE FIELD CORRECTION Manual Resuscitator**

## **Attention to Customer:**

Life Assist Inc.  
11277 Sunrise Park Dr  
Rancho Cordova CA 95742

Dear Device Customer,

## **Purpose of this letter:**

This letter is to inform you of a voluntary medical device field correction being implemented by SunMed Holdings LLC (also trading as Ventlab LLC). The products involved are manual resuscitators.

## **Reason for the Voluntary Recall:**

SunMed Holdings LLC (also trading as Ventlab LLC) is updating its IFU for the manual resuscitators to include a warning statement. Additionally, we are issuing a safety bulletin relative to the use of accessories (e.g., PEEP valve) on the exhalation port of the manual resuscitator which may increase the backward leak due to increased expiratory resistance and that monitoring of end tidal carbon dioxide is recommended in critically ill patients.

## **Risk to Health:**


A backwards leak present in the integrated manometer may increase with the use of accessories (e.g., PEEP valve) on the exhalation port due to increased expiratory resistance.

A remote probability of medically reversible or transient adverse health consequence for premature infants, newborns, patients with traumatic brain injury, post-cardiac arrest patients, and stroke patients. Monitoring of end tidal carbon dioxide is recommended in critically ill patients.

## **How to recognize the affected device:**

All lots manufactured from 9/21/2018 to present with duckbill-style patient valve with integrated manometer are included in the scope of this correction. The below image will assist in identification of affected product.

**Affected product image examples:**

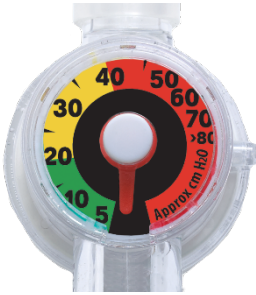

\*\*



**\*\*NOTE:** Image is an example of a manual resuscitator. Other configurations and colors are not shown.

**Actions to be taken by the Customer/User:**

- Monitoring of end tidal carbon dioxide is recommended in critically ill patients.
- Disseminate this information to all users.
- Complete the Manual Resuscitator Field Correction Response Form and send to [ProductQuality@myAirLife.com](mailto:ProductQuality@myAirLife.com)

**Product and Distribution Information:**

This table is not limited to the information listed below; please see Attachment A for a table that provides the product description and SKU of the affected products.

Product and Distribution Information Table					
Product Name	Manufacturer's Product Number	Lot Number	Manufacturing/Distribution Dates	Expiration Date (MM/DD/YYYY)	Quantity
See Attachment A	See Attachment A	All lots manufactured from 9/21/2018 to present	All Manufacturing Dates 9/21/2018 to present	10/11/2028	All Quantities

**Type of Action by the Company:**

Ventlab LLC is updating its IFU for the manual resuscitators to include a warning statement. Additionally, we are issuing a safety bulletin relative to the use of accessories (e.g., PEEP valve) on the exhalation port of the manual resuscitator which may increase the backward leak due to increased expiratory resistance and that monitoring of end tidal carbon dioxide is recommended in critically ill patients.

**OTHER INFORMATION:**

- Excluded from this voluntary field correction are manual resuscitators made with an umbrella style patient valve and all configurations without an integrated manometer.

- Attachments:
  - *Manual Resuscitator Field Correction Response Form*
  - *Affected Product List (Attachment A)*
- Should you have any questions, feel free to reach out to your local AirLife Territory Manager, Customer Service at 1.800.433.2797 or [ProductQuality@myAirLife.com](mailto:ProductQuality@myAirLife.com)

Thanks,

A handwritten signature in black ink, appearing to read 'Rob Yamashita', with a stylized flourish at the end.

Rob Yamashita  
VP of Regulatory Affairs

Adverse reactions or quality problems experienced with the use of this product may be reported to the FDA's MedWatch Adverse Event Reporting program either online, by regular mail or by fax.

**Immediate Action Requested**  
***Manual Resuscitator Field Correction Response Form***

- I have read and understand the notification letter sent October 11, 2023.
- I have further questions and wish to speak to someone directly.

Contact Name: \_\_\_\_\_

Title: \_\_\_\_\_

Facility Name: \_\_\_\_\_

Address: \_\_\_\_\_

City/State/Zip Code: \_\_\_\_\_

Telephone Number: \_\_\_\_\_

Email: \_\_\_\_\_

PLEASE SEND COMPLETED RESPONSE FORM TO: [ProductQuality@myAirLife.com](mailto:ProductQuality@myAirLife.com)

## ATTACHMENT A: AFFECTED PRODUCT LIST

ITEM NUMBER	ITEM DESCRIPTION
AF5140MB	AIRFLOW ADULT RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, BAG RESERVOIR, AND MASK
AF1140MB	AIRFLOW ADULT RESUSCITATOR WITH MASK, MANOMETER, BACTERIAL/VIRAL FILTER AND BAG
AF1140MB-P	AIRFLOW ADULT RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, BAG RESERVOIR, MASK, AND PEEP VALVE
AF5140MBP	AIRFLOW ADULT RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, BAG RESERVOIR, MASK, AND PEEP VALVE
AF3140MB	AIRFLOW INFANT RESUSCITATOR WITH INTEGRATED MANOMETER, 40CM POP-OFF, B/V FILTER, BAG RESERVOIR, AND MASK
AF5140MB-PW	AIRFLOW ADULT RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, BAG RESERVOIR, MASK, AND ATTACHED PEEP VALVE
VN5000MBPF	V-CARE SMALL ADULT RESUSCITATOR WITH BAG RESERVOIR, PEEP VALVE, B/V FILTER & MASK
AF2142MB	AIRFLOW PEDIATRIC RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, 40CM POP-OFF, BAG RESERVOIR, AND MASK
AF2140MB	AIRFLOW PEDIATRIC RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, BAG RESERVOIR, AND MASK
AF5140MB-2	AIRFLOW ADULT RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, BAG RESERVOIR, AND SIZE 5 & 6 MASKS
AF5140MBPF-G10	AIRFLOW ADULT RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, BAG RESERVOIR, MASK, PEEP VALVE, ADDITIONAL B/V FILTER, AND 100 MM AIRWAY
AF2142MB-3	AIRFLOW PEDIATRIC RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, 40CM POP-OFF, BAG RESERVOIR, AND SIZE 3 & 4 MASKS
AF5140MC	AIRFLOW ADULT RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, CORRUGATED TUBING, AND MASK
AF4140MBP	AIRFLOW CHILD RESUSCITATOR WITH INTEGRATED MANOMETER, 40CM POP-OFF, B/V FILTER, BAG RESERVOIR, MASK, AND PEEP VALVE
AF3140MB-2	AIRFLOW INFANT RESUSCITATOR WITH INTEGRATED MANOMETER, 40CM POP-OFF, B/V FILTER, BAG RESERVOIR, AND SIZE 1 & 2 MASKS
SS3200MB-2	SAFESPOT INFANT RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, 40CM H2O POP-OFF & 2 MASKS
AF5142MB	AIRFLOW ADULT RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, 40CM POP-OFF, BAG RESERVOIR, AND MASK
AF3140MB-P	AIRFLOW INFANT RESUSCITATOR WITH INTEGRATED MANOMETER, 40CM POP-OFF, B/V FILTER, BAG RESERVOIR, MASK, AND PEEP VALVE
BT2016	BREATHTECH INFANT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF & MASK
AF3100MBP-D12	AIRFLOW INFANT RESUSCITATOR WITH INTEGRATED MANOMETER, 40CM POP-OFF, BAG RESERVOIR, SIZE 1 & 2 MASKS, AND PEEP VALVE

BVMB510S-F	RESCUMED ADULT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, MASK, HAND STRAP & B/V FILTER
AF3140MCPF	AIRFLOW INFANT RESUSCITATOR WITH INTEGRATED MANOMETER, 40CM POP-OFF, B/V FILTER, CORRUGATED TUBING, MASK, PEEP VALVE, AND ADDITIONAL B/V FILTER
AF2140MBP	AIRFLOW PEDIATRIC RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, BAG RESERVOIR, MASK, AND PEEP VALVE
AF5140MBS-I	AIRFLOW ADULT RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, BAG RESERVOIR, MASK, HAND STRAP, AND FENEM
AF5140MB-I	AIRFLOW ADULT RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, BAG RESERVOIR, MASK, AND FENEM
AF2142OBNS	AIRFLOW PEDIATRIC RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, 40CM POP-OFF, BAG RESERVOIR, AND FLEX NECK, AND HAND STRAP
AF2142MBP	AIRFLOW PEDIATRIC RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, 40CM POP-OFF, BAG RESERVOIR, MASK, AND PEEP VALVE
AF5140MB-D6	AIRFLOW ADULT RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, BAG RESERVOIR, AND SIZE 6 MASK
AF2142MC-3	AIRFLOW PEDIATRIC RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, 40CM POP-OFF, CORRUGATED TUBING, AND SIZE 1, 3 & 4 MASKS
BT2023	BREATHTECH INFANT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF, PEEP VALVE & 2 MASKS
VN3100MB	V-CARE INFANT RESUSCITATOR WITH BAG RESERVOIR, MANOMETER & MASK
VN3100MB-2	V-CARE INFANT RESUSCITATOR WITH BAG RESERVOIR, MANOMETER & 2 MASKS
AF3100OB	AIRFLOW INFANT RESUSCITATOR WITH INTEGRATED MANOMETER, 40CM POP-OFF, AND BAG RESERVOIR
AF5140MBP-I	AIRFLOW ADULT RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, BAG RESERVOIR, MASK, PEEP VALVE, AND FENEM
AF4140MB	AIRFLOW CHILD RESUSCITATOR WITH INTEGRATED MANOMETER, 40CM POP-OFF, B/V FILTER, BAG RESERVOIR, AND MASK
AF1140MB-I	AIRFLOW ADULT RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, BAG RESERVOIR, MASK, AND FENEM
AF4100MBP	AIRFLOW CHILD RESUSCITATOR WITH INTEGRATED MANOMETER, 40CM POP-OFF, BAG RESERVOIR, MASK, AND PEEP VALVE
VN3100MCP-2	V-CARE INFANT RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, MANOMETER, PEEP VALVE & 2 MASKS
VN3100MC	V-CARE INFANT RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, MANOMETER & MASK
AF1140MBS	AIRFLOW ADULT RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, BAG RESERVOIR, MASK, AND HAND STRAP
AF3100MBP	AIRFLOW INFANT RESUSCITATOR WITH INTEGRATED MANOMETER, 40CM POP-OFF, BAG RESERVOIR, MASK, AND PEEP VALVE
BT4018	BREATHTECH PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF & MASK

BT2037	BREATHTECH INFANT RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF, PEEP VALVE & 2 MASKS
AF2142MBP-2	AIRFLOW PEDIATRIC RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, 40CM POP-OFF, BAG RESERVOIR, SIZE 3 & 4 MASKS, AND PEEP VALVE
AF1100MB	AIRFLOW ADULT RESUSCITATOR WITH INTEGRATED MANOMETER, BAG RESERVOIR, AND MASK
AF2142MB-2	AIRFLOW PEDIATRIC RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, 40CM POP-OFF, BAG RESERVOIR, AND SIZE 3 & 4 MASKS
AF3140MBP-2	AIRFLOW INFANT RESUSCITATOR WITH INTEGRATED MANOMETER, 40CM POP-OFF, B/V FILTER, BAG RESERVOIR, SIZE 1 & 2 MASKS, AND PEEP VALVE
VN3104MCFB-M01	V-CARE INFANT RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, MANOMETER, 25CM H2O POP-OFF, 2 MASKS & B/V FILTER
AF3144MCP-M1M00	AIRFLOW INFANT RESUSCITATOR WITH INTEGRATED MANOMETER, 25CM POP-OFF, B/V FILTER, CORRUGATED TUBING, SIZE 1 AND 00 MASKS, AND PEEP VALVE
VN5000MXP-K	V-CARE SMALL ADULT RESUSCITATOR WITH COLLAPSIBLE ACCUMULATOR, PEEP VALVE, MASK & STAT-CHECK
AF2140MCPF	AIRFLOW PEDIATRIC RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, CORRUGATED TUBING, MASK, PEEP VALVE, AND ADDITIONAL B/V FILTER
AF5102MBP	AIRFLOW ADULT RESUSCITATOR WITH MASK, BAG RESERVOIR, MANOMETER, 40CM POP-OFF & PEEP VALVE
AF3140MCP	AIRFLOW INFANT RESUSCITATOR WITH INTEGRATED MANOMETER, 40CM POP-OFF, B/V FILTER, CORRUGATED TUBING, MASK, AND PEEP VALVE
AF2140MB-PW	AIRFLOW PEDIATRIC RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, BAG RESERVOIR, MASK, AND ATTACHED PEEP VALVE
AF3100MB	AIRFLOW INFANT RESUSCITATOR WITH INTEGRATED MANOMETER, 40CM POP-OFF, BAG RESERVOIR, AND MASK
AF2142MCP-D4	AIRFLOW PEDIATRIC RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, 40CM POP-OFF, CORRUGATED TUBING, SIZE 4 MASK, AND PEEP VALVE
VNW3100MB-2	V-CARE INFANT RESUSCITATOR WITH BAG RESERVOIR, MANOMETER & 2 MASKS
VN3100MBPW-D2	V-CARE INFANT RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, PEEP VALVE & MASK
SS3200MB	SAFESPOT INFANT RESUSCITATOR WITH INTEGRATED MANOMETER, BAG RESERVOIR, 40CM H2O POP-OFF, AND MASK
AF2142MB-PW	AIRFLOW PEDIATRIC RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, 40CM POP-OFF, BAG RESERVOIR, MASK, AND ATTACHED PEEP VALVE
AF3100MBP-M01	AIRFLOW INFANT RESUSCITATOR WITH INTEGRATED MANOMETER, 40CM POP-OFF, BAG RESERVOIR, SIZE 0 & 1 MASKS, AND PEEP VALVE
AF1140MC	AIRFLOW ADULT RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, CORRUGATED TUBING, AND MASK
AF1140MBP-2	AIRFLOW ADULT RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, BAG RESERVOIR, SIZE 4 & 5 MASKS, AND PEEP VALVE

BT4024	BREATHTECH PEDIATRIC RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF, PEEP VALVE & MASK
AF3140MB-M1	AIRFLOW INFANT RESUSCITATOR WITH INTEGRATED MANOMETER, 40CM POP-OFF, B/V FILTER, BAG RESERVOIR, AND SIZE 1 MASK
AF2140MBPW-D23	AIRFLOW PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, FILTER, D2 & D3 MASKS & ATTACHED PEEP VALVE
AF5140MBP-G10	AIRFLOW ADULT RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, BAG RESERVOIR, MASK, PEEP VALVE, AND 100 MM AIRWAY
BT2026	BREATHTECH INFANT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF, PEEP VALVE & 2 MASKS
AF3140MB-PW	AIRFLOW INFANT RESUSCITATOR WITH INTEGRATED MANOMETER, 40CM POP-OFF, B/V FILTER, BAG RESERVOIR, MASK, AND ATTACHED PEEP VALVE
VN3100MC-2	V-CARE INFANT RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, MANOMETER & 2 MASKS
AF2100OB	AIRFLOW PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR & MANOMETER
AF1140MB-K	AIRFLOW ADULT RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, BAG RESERVOIR, MASK, AND STAT-CHECK
AF1140MB-PW	AIRFLOW ADULT RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, BAG RESERVOIR, MASK, AND ATTACHED PEEP VALVE
AF1140MB-X	AIRFLOW ADULT RESUSCITATOR W/ INFLATABLE BAG RESERVOIR, MANO, FIL
AF5140MBP-D6	AIRFLOW SMALL ADULT RESUSCITATOR WITH MANOMETER, FILTER, D6 MASK, BAG RESERVOIR & PEEP VALVE
AF3100MB-M1	AIRFLOW INFANT RESUSCITATOR WITH INTEGRATED MANOMETER, 40CM POP-OFF, BAG RESERVOIR, AND SIZE 1 MASK
VN2102MBF	PEDI BAG, MASK W/ INFLAT BAG RES, POP-OFF, MANOMETER & FILTER
AF4100MB	AIRFLOW CHILD RESUSCITATOR WITH INTEGRATED MANOMETER, 40CM POP-OFF, BAG RESERVOIR, AND MASK
BT2027	BREATHTECH INFANT RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF & 2 MASKS
AF3140MC	AIRFLOW INFANT RESUSCITATOR WITH INTEGRATED MANOMETER, 40CM POP-OFF, B/V FILTER, CORRUGATED TUBING, AND MASK
AF3100MCP-2	AIRFLOW INFANT RESUSCITATOR WITH CORRUGATED TUBING, INTEGRATED MANOMETER, PEEP VALVE & 2 MASKS
BT2032	BREATHTECH INFANT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF & 2 MASKS
AF5140MBS	AIRFLOW ADULT RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, BAG RESERVOIR, MASK, AND HAND STRAP
AF3100MB-X	AIRFLOW INFANT RESUSCITATOR WITH INTEGRATED MANOMETER, 40CM POP-OFF, BAG RESERVOIR, MASK, AND MAXCAP
AF4140MB-I	AIRFLOW CHILD RESUSCITATOR WITH INTEGRATED MANOMETER, 40CM POP-OFF, B/V FILTER, BAG RESERVOIR, MASK, AND FENEM
AF2102MC-PT	AIRFLOW PEDIATRIC RESUSCITATOR WITH INTEGRATED MANOMETER, 40CM POP-OFF, CORRUGATED TUBING, MASK, PEEP VALVE, AND 14' TUBING



AF5100MB	AIRFLOW ADULT RESUSCITATOR WITH INTEGRATED MANOMETER, BAG RESERVOIR, AND MASK
AF2142OB	AIRFLOW PEDIATRIC RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, 40CM POP-OFF, AND BAG RESERVOIR
SS3200MB-MMC	SAFESPOT INFANT RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, 40CM H2O POP-OFF, MASK & EXHALATION PORT STICKER
AF4140MB-PI	AIRFLOW CHILD RESUSCITATOR WITH INTEGRATED MANOMETER, 40CM POP-OFF, B/V FILTER, BAG RESERVOIR, MASK, PEEP VALVE, AND FENEM
VN2100MB	V-CARE PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, MANOMETER AND MASK
AF1100MB-N	AIRFLOW ADULT RESUSCITATOR WITH MASK, BAG, MANOMETER AND FLEX NECK
SS3200MB-I	SAFESPOT INFANT RESUSCITATOR WITH INTEGRATED MANOMETER, BAG RESERVOIR, 40CM H2O POP-OFF, MASK, AND FENEM
VN2102MC	V-CARE PEDIATRIC RESUSCITATOR WITH CORRUGATED TUBING, MANOMETER, 40CM H2O POP-OFF VALVE & MASK
BT2020	BREATHTECH INFANT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF, PEEP VALVE & MASK
AF1140MB-D5	AIRFLOW ADULT RESUSCITATOR WITH D5 MASK, MANOMETER, BACTERIAL/VIRAL FILTER, AND BAG
BVMB710-F	RESCUMED INFANT RESUSCITATOR WITH INTEGRATED MANOMETER AND B/V FILTER
VN5000MB	V-CARE SMALL ADULT RESUSCITATOR WITH BAG RESERVOIR & MASK
AF2100MB	AIRFLOW PEDIATRIC RESUSCITATOR WITH INTEGRATED MANOMETER, BAG RESERVOIR, AND MASK
AF2100MB-MMC	AIRFLOW PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, MASK & EXHALATION PORT STICKER
AF3100MB-2	AIRFLOW INFANT RESUSCITATOR WITH INTEGRATED MANOMETER, 40CM POP-OFF, BAG RESERVOIR, AND SIZE 1 & 2 MASKS
VN3100OB	V-CARE INFANT RESUSCITATOR WITH BAG RESERVOIR & MANOMETER
VN5000MBPF-I	SMALL ADULT RESUS W/MASK, BAG RESERV, FILTER, PEEP AND FENEM
AF1140MBP-T	AIRFLOW ADULT RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, BAG RESERVOIR, MASK, PEEP VALVE, AND 14' TUBING
AF2102MBP	AIRFLOW PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, MASK & PEEP VALVE
VN3100MBF	V-CARE INFANT RESUSCITATOR WITH MANOMETER, MASK, BAG RESERVOIR & FILTER
AF1140MB-R	AIRFLOW ADULT RESUSCITATOR WITH MASK, MANOMETER, BACTERIAL/VIRAL FILTER, BAG AND ACE
BVMB510-F	RESCUMED ADULT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, MASK & B/V FILTER
VN3100MB-D1	V-CARE INFANT RESUSCITATOR WITH BAG RESERVOIR, MANOMETER & MASK
AF5140MBPWT5	AIRFLOW SMALL ADULT RESUSCITATOR WITH MANOMETER, FILTER, D5 MASK, BAG RESERVOIR, PEEP VALVE ATTACHED & 14' TUBING

AF3100MBP-MOD1	AIRFLOW INFANT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF, PEEP VALVE, SIZE 0 & D1 MASKS
AF5142MBP	AIRFLOW ADULT RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, 40CM POP-OFF, BAG RESERVOIR, MASK, AND PEEP VALVE
AF1142MBP	AIRFLOW ADULT RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, 40CM POP-OFF, BAG RESERVOIR, MASK, AND PEEP VALVE
SS3200MC	SAFESPOT INFANT RESUSCITATOR WITH INTEGRATED MANOMETER, CORRUGATED TUBING, 40CM H2O POP-OFF, AND MASK
AF5100MB-PW	AIRFLOW SMALL ADULT RESUSCITATOR WITH MANOMETER, MASK, PEEP VALVE ATTACHED, BAG RESERVOIR
VN2102MB	V-CARE PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, 40CM H2O POP-OFF VALVE AND MASK
AF1140MBP-K	AIRFLOW ADULT RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, BAG RESERVOIR, MASK, PEEP VALVE, AND STAT-CHECK
AF2142MB-D3	AIRFLOW PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, FILTER, 40CM H2O POP-OFF & D3 MASK
SS3200MB-D1	SAFESPOT INFANT RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, 40CM H2O POP-OFF & MASK
VN5000MX	V-CARE SMALL ADULT RESUSCITATOR WITH COLLAPSIBLE ACCUMULATOR & MASK
AF1140MCP	AIRFLOW ADULT RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, CORRUGATED TUBING, MASK, AND PEEP VALVE
AF3140MB-I	AIRFLOW INFANT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, B/V FILTER, MASK & FENEM
AF3140MB-D12I	AIRFLOW INFANT RESUSCITATOR WITH INTEGRATED MANOMETER, 40CM POP-OFF, B/V FILTER, BAG RESERVOIR, SIZE 1 & 2 MASKS, AND FENEM
SS3204MB	SAFESPOT INFANT RESUSCITATOR WITH INTEGRATED MANOMETER, BAG RESERVOIR, 25CM H2O POP-OFF, AND MASK
BVMB810S-F	RESCUMED PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, MASK, HAND STRAP & B/V FILTER
AF4100OB	AIRFLOW CHILD RESUSCITATOR WITH MANOMETER AND BAG RESERVOIR
AF5140MB-F	AIRFLOW ADULT RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, BAG RESERVOIR, MASK AND ADDITIONAL B/V FILTER
BVMB810	RESCUMED PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER & MASK
AF2100MCP	AIRFLOW PEDIATRIC RESUSCITATOR WITH CORRUGATED TUBING, MANOMETER, MASK & PEEP VALVE
SC8121B	STAT-CHECK PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF & MASK
AF3100MB-MO	AIRFLOW INFANT RESUSCITATOR WITH INTEGRATED MANOMETER, 40CM POP-OFF, BAG RESERVOIR, AND SIZE 0 MASK
AF2102MB	AIRFLOW PEDIATRIC RESUSCITATOR WITH INTEGRATED MANOMETER, 40CM POP-OFF, BAG RESERVOIR, AND MASK

VN5102MC	V-CARE SMALL ADULT RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, MANOMETER, 40CM H2O POP-OFF & MASK
AF2140MC	AIRFLOW PEDIATRIC RESUSCITATOR WITH CORRUGATED TUBING, MANOMETER, FILTER & MASK
AF5140MBS-R	AIRFLOW SMALL ADULT RESUSCITATOR WITH MANOMETER, FILTER, MASK, BAG RESERVOIR, HAND STRAP & CO2 ATTACHED
VN4100MB	V-CARE CHILD RESUSCITATOR WITH BAG RESERVOIR, MANOMETER & MASK
AF3144MB	AIRFLOW INFANT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, 25CM H2O POP-OFF, B/V FILTER & MASK
SS3204MB-D12	SAFESPOT INFANT RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, 25CM H2O POP-OFF & 2 MASKS
AF3100MB-PTS1	AIRFLOW INFANT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, PEEP VALVE, 14' O2 TUBING & MASK
AF3100OC	AIRFLOW INFANT RESUSCITATOR WITH CORRUGATED TUBING
BT2220-2	BREATHTECH INFANT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF, PEEP VALVE & 2 MASKS
AF5140MC-P	AIRFLOW ADULT RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, CORRUGATED TUBING, MASK, AND PEEP VALVE
9105HZ	ADULT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, PEEP VALVE, MASK, HAND STRAP & B/V FILTER
AF3140MC-MOD12	AIRFLOW INFANT RESUSCITATOR WITH CORRUGATED TUBING, MANOMETER, FILTER, 40CM H2O POP-OFF & SIZE 0, 1 & 2 MASKS
BT2029	BREATHTECH INFANT RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF, PEEP VALVE & 2 MASKS
AF2144MB	AIRFLOW PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, FILTER, 25CM H2O POP-OFF & MASK
AF3100MB-PW	AIRFLOW INFANT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, PEEP VALVE & MASK
VN4100MCP	V-CARE CHILD RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, MANOMETER, PEEP VALVE & MASK
VN5000MXP	V-CARE SMALL ADULT RESUSCITATOR WITH COLLAPSIBLE ACCUMULATOR, PEEP VALVE & MASK
BT4040	BREATHTECH PEDIATRIC RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, 40CM H2O POP-OFF, PEEP VALVE & 2 MASKS
6118HZ	HORIZON INFANT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF & MASK
VN3100MB-I	V-CARE INFANT RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, MASK & FENEM
AF2100MB-3	AIRFLOW PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, MANOMETER & 3 MASKS
AF5142MB-D235	AIRFLOW ADULT RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, 40CM POP-OFF, BAG RESERVOIR, AND SIZE 2, 3 & 5 MASKS
BT2048	BREATHTECH INFANT RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF, PEEP VALVE & 2 MASKS

VN3100MC-D1	INFANT BAG,D1MSK W/CORR TUB RES&BUILT-IN MANOMETER
BVMB810-F	RESCUMED PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, MASK & B/V FILTER
SS3200MBP-2	SAFESPOT INFANT RESUSCITATOR WITH INTEGRATED MANOMETER, BAG RESERVOIR, 40CM H2O POP-OFF, AND SIZE 1 & 2 MASKS
BT4013F	PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF, 3 MASKS & B/V FILTER
BT2416	BREATHTECH INFANT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, 25CM H2O POP-OFF, MASK & B/V FILTER
VN3100MB-M01	V-CARE INFANT RESUSCITATOR WITH BAG RESERVOIR, MANOMETER & 2 MASKS
AF2102MBPW-D4	AIRFLOW PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, 40CM H2O POP-OFF, D4 MASK & ATTACHED PEEP VALVE
VN5000MC	V-CARE SMALL ADULT RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR & MASK
VN2100MC	PEDI BAG, MASKW/CORR O2RES & BUILT-IN MANOMETER
VN2100MCP	PEDI BAG, MASK W/CORR TUB O2 RES, PEEP & MANOMETER
AF5140MBPW-D5	AIRFLOW SMALL ADULT RESUSCITATOR WITH MANOMETER, FILTER, BAG RESERVOIR, D5 MASK & PEEP VALVE ATTACHED
VN3106MC-D2V	INF RESUS W/MAN.,D2 MASK, CORR RESERV.
SS3200MBP-2I	SAFESPOT INFANT RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, 40CM H2O POP-OFF, PEEP VALVE, 2 MASKS & FENEM
AF2140MB-2	AIRFLOW PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, FILTER & 2 MASKS
AF2102MC	AIRFLOW PEDIATRIC RESUSCITATOR WITH CORRUGATED TUBING, MANOMETER, 40CM H2O POP-OFF & MASK
VN5000MC-N	V-CARE SMALL ADULT RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, MASK & FLEX NECK
AF2142MB-K	AIRFLOW PEDIATRIC RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, 40CM POP-OFF, BAG RESERVOIR, MASK, AND STAT-CHECK
VN3100MB-PW2	INF W/MAN, PEEP ATTACHED,2 MSK
VN3100MCP	V-CARE INFANT RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, MANOMETER, PEEP VALVE & MASK
8116HZ	HORIZON PEDIATRIC RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF, PEEP VALVE & 3 MASKS
SC9101B	STAT-CHECK ADULT RESUSCITATOR WITH BAG RESERVOIR, MANOMETER & MASK
AF2102MCP	AIRFLOW PEDIATRIC RESUSCITATOR WITH INTEGRATED MANOMETER, 40CM POP-OFF, CORRUGATED TUBING, MASK, AND PEEP VALVE
VN2102OBP	PEDI BAG W/INFLAT BAG RES,PEEP,POP-OFF W/MANOMETER
VN3100MB-M00-0	INFANT RESUS W/M00 & M0 MASKS
VN3100MX-F	V-CARE INFANT RESUSCITATOR WITH COLLAPSIBLE ACCUMULATOR, MANOMETER, MASK & B/V FILTER

AF5140MBS-P	AIRFLOW SMALL ADULT RESUSCITATOR WITH MANOMETER, FILTER, MASK, BAG RESERVOIR, HAND STRAP & PEEP VALVE
AF4140MC	AIRFLOW CHILD RESUSCITATOR WITH MANOMETER, FILTER, MASK, AND CORRUGATED TUBING
VN2102MX	PEDI BAG, MASK W/COLLAP ACC & BUILT-IN MANOMETER
AF1100MBP	AIRFLOW ADULT RESUSCITATOR WITH MASK, MANOMETER, PEEP VALVE AND BAG
AF2100MC	AIRFLOW PEDIATRIC RESUSCITATOR WITH CORRUGATED TUBING, MANOMETER & MASK
BVMB510-10	RESCUMED ADULT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER & MASK
VN5000MX-N	V-CARE SMALL ADULT RESUSCITATOR WITH COLLAPSIBLE ACCUMULATOR, MASK & FLEX NECK
BT4022	BREATHTECH PEDIATRIC RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, INTEGRATED MANOMETER & MASK
AFN5140MB	AIRFLOW SMALL ADULT RESUSCITATOR WITH MANOMETER, FILTER, MASK & BAG RESERVOIR
BT4015	BREATHTECH PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF, PEEP VALVE & MASK
6009HZ	INFANT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF & MASK
6013HZ	INFANT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF, PEEP VALVE & MASK
6019HZ	INFANT RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF & MASK
6106HZ	INFANT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF & MASK
6107HZ	INFANT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF & PEEP VALVE
6112HZ	INFANT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, 25CM H2O POP-OFF & MASK
6116HZ	INFANT RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF, PEEP VALVE & MASK
6117HZ	INFANT RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF, PEEP VALVE & 2 MASKS
6123HZ	INFANT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, 25CM H2O POP-OFF, PEEP VALVE & MASK
6125HZ	INFANT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, 25CM H2O POP-OFF, PEEP VALVE & MASK
6200HZ	INFANT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, 25CM H2O POP-OFF, PEEP VALVE & 2 MASKS
8003HZ	HORIZON PEDIATRIC RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF, PEEP VALVE, MASK & B/V FILTER

8004HZ	PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, PEEP VALVE & MASK
8010HZ	HORIZON PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF, PEEP VALVE & MASK
8105HZ	PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, PEEP VALVE, MASK, HAND STRAP & B/V FILTER
8106HZ	PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER & MASK
8107HZ	HORIZON PEDIATRIC RESUSCIRATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF, MASK & B/V FILTER
8109HZ	HORIZON PEDIATRIC RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF & PEEP VALVE
8113HZ	HORIZON PEDIATRIC RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, INTEGRATED MANOMETER, 25CM H2O POP-OFF & MASK
8115HZ	PEDIATRIC RESUSCITATOR WITHBAG RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF & MASK
8119HZ	PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, 25CM H2O POP-OFF, PEEP VALVE & MASK
9002HZ	ADULT RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, INTEGRATED MANOMETER, PEEP VALVE & MASK
9006HZ	ADULT RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF, PEEP VALVE, MASK & B/V FILTER
9100HZ	HORIZON ADULT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, MASK, HAND STRAP & B/V FILTER
9106HZ	ADULT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER & MASK
9108HZ	ADULT RESUSCITATOR WITH COLLAPSIBLE ACCUMULATOR, INTEGRATED MANOMETER, PEEP VALVE & MASK
9110HZ	ADULT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER & PEEP VALVE
9111HZ	ADULT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER MASK & B/V FILTER
9120HZ	ADULT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, 25CM H2O POP-OFF, PEEP VALVE & MASK
9122HZ	ADULT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF, PEEP VALVE & MASK
AF1000MBN-20	AIRFLOW ADULT RESUSCITATOR WITH MASK, BAG RESERVOIR, FLEX NECK AND 20' TUBING
AF1100MB-MMC	AIRFLOW ADULT RESUSCITATOR WITH MASK, MANOMETER, BAG, AND EXHALATION PORT STICKER
AF1100MB-PWG1	AIRFLOW ADULT RESUSCITATOR WITH MASK, MANOMETER, BAG, PEEP ATTACHED AND GUEDEL AIRWAY
AF1100MBS	AIRFLOW ADULT RESUSCITATOR WITH MASK, MANOMETER, BAG AND STRAP

AF1100MC	AIRFLOW ADULT RESUSCITATOR WITH MANOMETER AND CORRUGATED TUBING
AF1100MC-P	AIRFLOW ADULT RESUSCITATOR WITH MASK, CORRUGATED TUBING, MANOMETER AND PEEP VALVE
AF1140MBC-N	AIRFLOW ADULT RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, BAG RESERVOIR, MASK, STRAIGHT O2 CONNECTOR, AND FLEX NECK
AF1140MB-E	AIRFLOW ADULT RESUSCITATOR WITH MASK, MANOMETER, BACTERIAL/VIRAL FILTER AND EXTENDED BAG
AF1140MB-M5	AIRFLOW ADULT RESUSCITATOR WITH #5 BLOW MOLD MASK, INFLATABLE BAG RESUSCITATOR, MANOMETER AND BACTERIAL/VIRAL FILTER
AF1140MB-N	AIRFLOW ADULT RESUSCITATOR WITH MASK, MANOMETER, BACTERIAL/VIRAL FILTER AND FLEX NECK
AF1140MBP-M5	AIRFLOW ADULT RESUSCITATOR WITH MANOMETER, FILTER, M5 MASK, BAG & PEEP VALVE
AF1140MBP-TM5	AIRFLOW ADULT RESUSCITATOR WITH M5 MASK, MANOMETER, BACTERIAL/VIRAL FILTER, BAG, PEEP VALVE AND 14 FT TUBING
AF1140MBS-K	AIRFLOW ADULT RESUSCITATOR WITH MASK, MANOMETER, BACTERIAL/VIRAL FILTER, BAG, STRAP AND STAT-CHECK
AF1140MBS-P	AIRFLOW ADULT RESUSCITATOR WITH MANOMETER, FILTER, MASK, BAG, STRAP & PEEP VALVE
AF1140MBSP-I	AIRFLOW ADULT RESUSCITATOR WITH MASK, MANOMETER, BACTERIAL/VIRAL FILTER, BAG, STRAP, PEEP VALVE AND FENEM
AF1140MB-T	AIRFLOW ADULT RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, BAG RESERVOIR, MASK, AND 14' TUBING
AF1140MB-TM5	AIRFLOW ADULT RESUSCITATOR WITH M5 MASK, INFLATABLE BAG RESUSCITATOR, BACTERIAL/VIRAL FILTER AND MANOMETER
AF1140MC-M5	AIRFLOW ADULT RESUSCITATOR WITH M5 MASK, CORRUGATED TUBE RESERVOIR, FILTER AND MANOMETER
AF1140OB	AIRFLOW ADULT RESUSCITATOR WITH MANOMETER, BACTERIAL/VIRAL FILTER AND BAG, NO MASK
AF1142MB	AIRFLOW ADULT RESUSCITATOR WITH MASK, MANOMETER, BACTERIAL/VIRAL FILTER, POP-OFF AND BAG
AF1142OB	AIRFLOW ADULT RESUSCITATOR WITH MANOMETER, FILTER, BAG RESERVOIR & POP-OFF
AF2100MB-D5	AIRFLOW PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, MANOMETER & D5 MASK
AF2100MB-PW	AIRFLOW PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, MASK & ATTACHED PEEP VALVE
AF2100MB-PWTS4	AIRFLOW PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, S4 MASK, PEEP VALVE ATTACHED & 14' TUBING
AF2102MBP-2	AIRFLOW PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, 40CM H2O POP-OFF, 2 MASKS & PEEP VALVE
AF2102MBP-M3	AIRFLOW PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, 40CM H2O POP-OFF, SIZE 3 MASK & PEEP VALVE

AF2102OB	AIRFLOW PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, MANOMETER & 40CM H2O POP-OFF
AF2140MB-D4	AIRFLOW PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, FILTER & D4 MASK
AF2140MB-D43	AIRFLOW PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, FILTER & SIZE D4 & D3 MASKS
AF2140MB-I	AIRFLOW PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, FILTER, MASK & FENEM
AF2140MB-K	AIRFLOW PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, FILTER, MASK & STAT-CHECK
AF2140MB-P2	AIRFLOW PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, FILTER, 2 MASKS & PEEP VALVE
AF2140MBP-K	AIRFLOW PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, FILTER, MASK, PEEP VALVE & STAT-CHECK
AF2140MB-S	AIRFLOW PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, FILTER, MASK & STRAP
AF2140MCP-D4	AIRFLOW PEDIATRIC RESUSCITATOR WITH CORRUGATED TUBING, MANOMETER, FILTER, D4 MASK & PEEP VALVE
AF2140MCP-K	AIRFLOW PEDIATRIC RESUSCITATOR WITH CORRUGATED TUBING, MANOMETER, FILTER, MASK, PEEP VALVE & STAT-CHECK
AF2140MC-PT	AIRFLOW PEDIATRIC RESUSCITATOR WITH CORRUGATED TUBING, MANOMETER, FILTER, MASK, PEEP VALVE & 14' TUBING
AF2140OB	AIRFLOW PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, MANOMETER & FILTER
AF2142MBP-224OT	AIRFLOW PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, FILTER, 40CM H2O POP-OFF, SIZE 2 & 4 MASKS, PEEP VALVE & 14' SALTER TUBING
AF2142MBS-I	AIRFLOW PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, FILTER, 40CM H2O POP-OFF, MASK, STRAP & FENEM
AF2142MC-D34	AIRFLOW PEDIATRIC RESUSCITATOR WITH CORRUGATED TUBING, MANOMETER, FILTER, 40CM H2O POP-OFF & SIZE D3 & D4 MASK
AF2142MC-M01	AIRFLOW PEDIATRIC RESUSCITATOR WITH CORRUGATED TUBING, MANOMETER, 40CM H2O POP-OFF & SIZE 0 & 1 MASKS
AF2142MCP	AIRFLOW PEDIATRIC RESUSCITATOR WITH CORRUGATED TUBING, MANOMETER, FILTER, PEEP VALVE, 40CM H2O POP-OFF & MASK
AF2242MBP	AIRFLOW PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, FILTER, 40CM H2O POP-OFF, MASK & PEEP VALVE
AF3100MB-M01	AIRFLOW INFANT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, SIZE 0 & 1 MASKS
AF3100MB-M0-D1	AIRFLOW INFANT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, SIZE 0 & D1 MASKS
AF3100MBP-2	AIRFLOW INFANT RESUSCITATOR WITH INTEGRATED MANOMETER, 40CM POP-OFF, BAG RESERVOIR, SIZE 1 & 2 MASKS , AND PEEP VALVE



AF3100MBP-M012	AIRFLOW INFANT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF, PEEP VALVE, SIZE 0, 1 & 2 MASKS
AF3100MC	AIRFLOW INFANT RESUSCITATOR WITH CORRUGATED TUBING, INTEGRATED MANOMETER & MASK
AF3100MC-2	AIRFLOW INFANT RESUSCITATOR WITH CORRUGATED TUBING, INTEGRATED MANOMETER & 2 MASKS
AF3100MCP	AIRFLOW INFANT RESUSCITATOR WITH INTEGRATED MANOMETER, 40CM POP-OFF, CORRUGATED TUBING, MASK, AND PEEP VALVE
AF3104MB-PW	AIRFLOW INFANT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, 25CM H2O POP-OFF, PEEP VALVE & MASK
AF3106MC-2	AIRFLOW INFANT RESUSCITATOR WITH CORRUGATED TUBING, INTEGRATED MANOMETER & MASK
AF3140MB-2B	AIRFLOW INFANT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, B/V FILTER, BLOW BY & 2 MASKS
AF3140MB-D1	AIRFLOW INFANT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, B/V FILTER & MASK
AF3140MB-D2	AIRFLOW INFANT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, B/V FILTER & D2 MASK
AF3140MB-K	AIRFLOW INFANT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, B/V FILTER, SIZE 0 MASK & STAT-CHECK
AF3140MB-M0	AIRFLOW INFANT RESUSCITATOR WITH INTEGRATED MANOMETER, 40CM POP-OFF, B/V FILTER, BAG RESERVOIR, AND SIZE 0 MASK
AF3140MB-M000	AIRFLOW INFANT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, B/V FILTER & 2 MASKS
AF3140MB-S1	AIRFLOW INFANT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, B/V FILTER & SIZE 1 MASK
AF3140MC-2B	AIRFLOW INFANT RESUSCITATOR WITH CORRUGATED TUBING, INTEGRATED MANOMETER, B/V FILTER, BLOW BY & 2 MASKS
AF3140MCP-K	AIRFLOW INFANT RESUSCITATOR WITH CORRUGATED TUBING, INTEGRATED MANOMETER, 40CM H2O POP-OFF, B/V FILTER, MASK & STAT-CHECK
AF3140OB	AIRFLOW INFANT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER & B/V FILTER
AF4100MCP	AIRFLOW CHILD RESUSCITATOR WITH CORRUGATE TUBING RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF, PEEP VALVE AND MASK
AF4140MB-D23	AIRFLOW CHILD RESUSCITATOR WITH MANOMETER, FILTER D2 & D3 MASK AND BAG
AF4140MCP-D3	AIRFLOW CHILD RESUSCITATOR WITH MANOMETER, FILTER, D3MASK, CORRUGATED TUBING & PEEP VALVE
AF4140OBP	AIRFLOW CHILD RESUSCITATOR WITH MANOMETER, FILTER, BAG RESERVOIR, PEEP VALVE, NO MASK
AF5100MB-D5	AIRFLOW SMALL ADULT RESUSCITATOR WITH MANOMETER, BAG RESERVOIR & D5 MASK
AF5100MC	AIRFLOW ADULT RESUSCITATOR WITH INTEGRATED MANOMETER, CORRUGATED TUBING, AND MASK

AF5102MB-N	AIRFLOW SMALL ADULT RESUSCITATOR WITH MASK, INFLATABLE BAG, 40CM POP-OFF, MANOMETER & FLEX NECK
AF5140MB-K	AIRFLOW SMALL ADULT RESUSCITATOR WITH MANOMETER, FILTER, MASK, BAG RESERVOIR & STAT-CHECK
AF5140MB-M5	AIRFLOW SMALL ADULT RESUSCITATOR WITH M5 MASK, INFLATABLE BAG RESERVOIR, FILTER AND MANOMETER
AF5140MBN	AIRFLOW ADULT RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, BAG RESERVOIR, MASK, AND FLEX NECK
AF5140MB-OT	AIRFLOW SMALL ADULT RESUSCITATOR WITH MANOMETER, FILTER, MASK, BAG RESERVOIR & 14' TUBING
AF5140MBP-D5	AIRFLOW SMALL ADULT RESUSCITATOR WITH MANOMETER, FILTER, D5 MASK, BAG, AND PEEP VALVE
AF5140MBP-M5	AIRFLOW SMALL ADULT RESUSCITATOR WITH MANOMETER, FILTER, M5 MASK, BAG RESERVOIR & PEEP VALVE
AF5140MB-PT	AIRFLOW SMALL ADULT RESUSCITATOR WITH MANOMETER, FILTER, MASK, BAG RESERVOIR, PEEP VALVE & 14' TUBING
AF5140MBPW-M5	AIRFLOW SMALL ADULT RESUSCITATOR WITH MANOMETER, FILTER, M5 MASK, BAG & PEEP VALVE
AF5140MBS-M5	AIRFLOW SMALL ADULT RESUSCITATOR, WITH #5 BLOW MOLD MASK, INFLATABLE BAG RESERVOIR, MANOMETER AND FILTER
AF5140MB-ST	AIRFLOW SMALL ADULT RESUSCITATOR WITH MANOMETER, FILTER, MASK, BAG RESERVOIR, STRAP & 14' TUBING
AF5140MB-T	AIRFLOW SMALL ADULT RESUSCITATOR WITH MANOMETER, FILTER, MASK, BAG RESERVOIR & 14' TUBING
AF5140MB-X	AIRFLOW ADULT RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, BAG RESERVOIR, MASK, AND MAXCAP
AF5140MC-D6	AIRFLOW SMALL ADULT RESUSCITATOR WITH CORRUGATED TUBING, INTEGRATED MANOMETER, B/V FILTER & MASK
AF5140MCP-K	AIRFLOW SMALL ADULT RESUSCITATOR WITH MANOMETER, FILTER, MASK, CORRUGATED TUBING, STAT CHECK CO2
AF5142MBP-M5	AIRFLOW SMALL ADULT RESUSCITATOR WITH MANOMETER, FILTER, 40CM POP-OFF, M5 MASK, BAG RESERVOIR & PEEP VALVE
AF5142OB	AIRFLOW ADULT RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, 40CM POP-OFF, AND BAG RESERVOIR
AFD5140MB-T	AIRFLOW SMALL ADULT RESUSCITATOR WITH MANOMETER, FILTER, MASK, BAG RESERVOIR & 14' TUBING
AFW1140MB	AIRFLOW ADULT RESUSCITATOR WITH MANOMETER, FILTER, MASK & BAG RESERVOIR
AFW2142MB-2	AIRFLOW PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, FILTER, 40CM H2O POP-OFF & 2 MASKS
AFW2142OB	AIRFLOW PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, FILTER & 40CM H2O POP-OFF
BT2014	BREATHTECH INFANT RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF & 2 MASKS

BT2015	BREATHTECH INFANT RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF & MASK
BT2034	BREATHTECH INFANT RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF, PEEP VALVE & MASK
BT2034W	BREATHTECH INFANT RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF, PEEP VALVE & MASK
BT2035	INFANT BAG, D1 MASK W/ INFLAT BAG RES & MANOMETER
BT2036	BREATHTECH INFANT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF & PEEP VALVE
BT2215	BREATHTECH INFANT RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, 40CM H2O POP-OFF & MASK
BT2216	BREATHTECH INFANT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF & MASK
BT2220	BREATHTECH INFANT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF, PEEP VALVE & MASK
BT2220F	BREATHTECH INFANT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF, PEEP VALVE, MASK & B/V FILTER
BT2234	BREATHTECH INFANT RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF, PEEP VALVE & MASK
BT2234FK	BREATHTECH INFANT RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF, PEEP VALVE, MASK, B/V FILTER & CO2 INDICATOR
BT2415	BREATHTECH INFANT RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, INTEGRATED MANOMETER, 25CM H2O POP-OFF, MASK & B/V FILTER
BT2415K	BREATHTECH INFANT RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, INTEGRATED MANOMETER, 25CM H2O POP-OFF, MASK & CO2 INDICATOR
BT3025	BREATHTECH CHILD RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, INTEGRATED MANOMETER, PEEP VALVE & 2 MASKS
BT3030	BREATHTECH CHILD RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF & 3 MASKS
BT3034	BREATHTECH CHILD RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF & 3 MASKS
BT4006-N	BREATHTECH PEDIATRIC RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, 40CM H2O POP-OFF, MASK & FLEX NECK
BT4013	BREATHTECH PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF & 2 MASKS
BT4016	BREATHTECH PEDIATRIC RESUSCITATOR WITH AER RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF, PEEP VALVE & MASK
BT4019	BREATHTECH PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER & MASK
BT4045	BREATHTECH PEDIATRIC RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF & MASK
BT5104P	BREATHTECH ADULT RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, INTEGRATED MANOMETER, PEEP VALVE & MASK

BT5107F	BREATHTECH ADULT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, PEEP VALVE, MASK & B/V FILTER
BVM510-F	RESCUMED ADULT RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, MASK & B/V FILTER
BVMB710	RESCUMED INFANT RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER AND 40CM POP-OFF VALVE
BVMB812	RESCUMED PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF & MASK
BVMB812-F	RESCUMED PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, INTEGRATED MANOMETER, 40CM H2O POP-OFF & MASK
HZI6000	INFANT RESUSCITATOR, MASK W/ INFLAT BAG RES, MANOMETER & 40CM POP
HZI8100	PEDI RESUSCITATOR, MASK W/ INFLAT BAG RES, MANOMETER & 40CM POP-O
HZI9000	ADULT RESUSCITATOR, MASK W/ INFLAT BAG RES, & MANOMETER (FOR INTE
HZI9108	ADULT RESUSCITATOR, MASK W/ INFLAT BAG RES, MANOMETER & STRAP
MC1000	RESUSCITATION KIT
PRO-1927	SAFE SPOT BAG, D1 MASK W/ INFLAT BAG RES, MANO, PEEP & 40CM
SC7101B	STAT-CHECK INFANT RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, 40CM H2O POP-OFF & MASK
SC7101B-M0	STAT-CHECK INFANT RESUSCITATOR WITH BAG RESERVOIR, MANOMETER & MASK
SS3200MBF	SAFESPOT INFANT RESUSCITATOR WITH INTEGRATED MANOMETER, BAG RESERVOIR, 40CM H2O POP-OFF, B/V FILTER, AND MASK
SS3200MBF-2	SAFESPOT INFANT RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, B/V FILTER & 2 MASKS
SS3200MBP	SAFESPOT INFANT RESUSCITATOR WITH INTEGRATED MANOMETER, BAG RESERVOIR, 40CM H2O POP-OFF, PEEP VALVE, AND MASK
SS3200MBP-M00	SAFESPOT INFANT RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, 40CM H2O POP-OFF, PEEP VALVE & MASK
SS3200MB-PW	SAFESPOT INFANT RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, 40CM H2O POP-OFF, PEEP VALVE & MASK
SS3200MB-PWS10	SAFESPOT INFANT RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, PEEP VALVE & 2 MASKS
SS3200MC-P2	SAFESPOT INFANT RESUSCITATOR WITH INTEGRATED MANOMETER, CORRUGATED TUBING, 40CM H2O POP-OFF, PEEP VALVE, AND SIZE 1 & 2 MASKS
SS3200OB	SAFESPOT INFANT RESUSCITATOR WITH INTEGRATED MANOMETER, BAG RESERVOIR, AND 40CM H2O POP-OFF
SS3200OBP	SAFESPOT INFANT RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, 40CM H2O POP-OFF & PEEP VALVE
SS3200OBPC-T	SAFESPOT INFANT RESUSCITATOR W/ BAG RESERVOIR, INTEGRATED MANOMETER, 40CM POP-OFF, PEEP, STRAIGHT CONNECTOR & 14' O2 TUBING
SS3204MC-P2	SAFESPOT INFANT RESUSCITATOR WITH INTEGRATED MANOMETER, BAG RESERVOIR, 25CM H2O POP-OFF, PEEP VALVE, AND SIZE 1 & 2 MASKS

SS3206MCP-S1	SAFESPOT INFANT RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, MANOMETER, PEEP VALVE & MASK
VN2100MBF	V-CARE PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, MASK AND FILTER
VN2100MBP	PEDI BAG, MASK W/INFLAT BAG, PEEP & BUILT-IN MANOMETER
VN2100MB-PW	V-CARE PEDIATRIC RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, PEEP VALVE ATTACHED AND MASK
VN2100MX-M123	V-CARE PEDIATRIC RESUSCITATOR WITH COLLAPSIBLE ACCUMULATOR, MANOMETER, SIZE 1, 2, AND 3 MASKS
VN2102MC-B	V-CARE PEDIATRIC RESUSCITATOR WITH CORRUGATED TUBING, MANOMETER, 40CM H2O POP-OFF VALVE, MASK & BLOW-BY
VN2102MCP	V-CARE PEDIATRIC RESUSCITATOR WITH CORRUGATED TUBING, MANOMETER, 40CM H2O POP-OFF VALVE, PEEP VALVE & MASK
VN2102MX-F	PEDI RESUS W/MAN, FILTER, POPOFF
VN3100MB-B	INF BAG W/MAN, MSK & BLOW-BY
VN3100MB-M1	INFANT W/MAN AND M1 MSK/NO VAL
VN3100MBP	V-CARE INFANT RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, PEEP VALVE & MASK
VN3100MBP-000	INFANT RESUS BAG W/MANOMETER, PEEP, SIZE 0 & 00 MASKS
VN3100MBP-2	V-CARE INFANT RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, PEEP VALVE & 2 MASKS
VN3100MBP-3M	V-CARE INFANT RESUSCITATOR WITH BAG RESERVOIR, MANOMETER & 3 MASKS
VN3100MBP-D1	V-CARE INFANT RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, PEEP VALVE & MASK
VN3100MBP-F	INFANT BAG, MAN, MSK, PEEP, FILTER
VN3100MC-2U	V-CARE INFANT RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, MANOMETER, 2 MASKS & GREEN FITS-ALL CONNECTOR
VN3100MC-B	INFANT BAG, MASK W/ CORR TUB RES, MAN & NO BLOW-BY
VN3100MXK-2	V-CARE INFANT RESUSCITATOR WITH COLLAPSIBLE ACCUMULATOR, MANOMETER, 2 MASKS & STAT-CHECK
VN3100OBP	INFANT BAG W/INFLAT BAG RES, PEEP & MANOMETER
VN3106MC-M1-2V	INF W/MAN., 12" CORR RESERV., M1 & M2 MASK & NEW STYLE O2 RESER
VN3106MC-M1V	V-CARE INFANT RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, MANOMETER, MASK & PORT TAIL ASSEMBLY
VN3106MCPF-M1V	V-CARE INFANT RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, MANOMETER, PEEP VALVE, MASK, B/V FILTER & PORT TAIL ASSEMBLY
VN4100MCFB-3	V-CARE CHILD RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, MANOMETER, 3 MASKS & B/V FILTER
VN5002MB	V-CARE SMALL ADULT RESUSCITATOR WITH BAG RESERVOIR, 40CM H2O POP-OFF & MASK
VN5100MB-F	V-CARE SMALL ADULT RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, B/V FILTER & MASK

VN5100MCP	V-CARE SMALL ADULT RESUSCITATOR WITH CORRUGATED TUBING RESERVOIR, MANOMETER, PEEP VALVE & MASK
VN5100MX-F	SM ADULT BAG, MASK W/COLLAP ACC, FILTER & MANOMETER
SS3204MBF-2	SAFESPOT INFANT RESUSCITATOR WITH BAG RESERVOIR, MANOMETER, B/V FILTER, 25CM H2O POP-OFF & 2 MASKS
AF2142MB-M5E	AIRFLOW PEDIATRIC RESUSCITATOR WITH INTEGRATED MANOMETER, B/V FILTER, 40CM POP-OFF, BAG RESERVOIR, GAS SAMPLING LINE, AND SIZE 5 MASK