


ITEM #

56307

MegaMover[®] Transport Chair

REV 05.01.19

Use Instructions: The MegaMover[®] Transport Chair is rated to transport up to 350 pounds (159 kg). The unit must be placed so handle support straps are to the outside of the patient. The Transport Chair is for use by professional staff for lifting and transporting/transferring patients. It is recommended to use 2-4 persons to provide safe transport/transfer, thus reducing the possibility of accident or injury. Grab handles in line with the people lifting on opposite side of the chair – do not "Cross-Handle". The MegaMover[®] Transport Chair is specifically for use when there is no need for spinal stabilization or traction of the patient being transported.

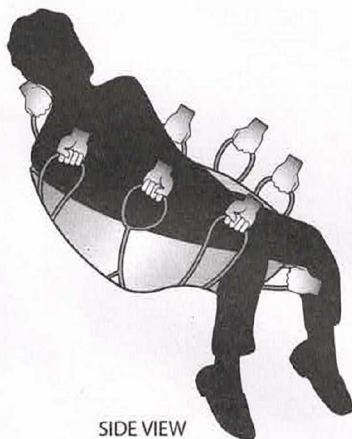

SIDE VIEW

Proper Grasping



**Improper Grasping
(Cross-Handle)**

The MegaMover[®] Transport Chair is intended for **limited use**. If the unit is damaged, frayed, cut or soiled replace with new unit.

- Clean soiled MegaMover[®] with damp cloth. Soap, detergent or disinfectant can be used.
- **Do Not** Machine Wash or Dry.
- Avoid contact with sharp objects.
- **Do Not** use if the unit is punctured, torn, frayed, or excessively worn.
- Only use handles for lifting.
- Use care to ensure grip is secure prior to lifting.
- **Do Not** store in contact with heat source greater than 200° F.
- **Avoid** dragging over rough surfaces.
- Use minimum of two (2) people to lift patient.
- **Do Not** attach to mechanical lifts.

Made In China

