



Nasopharyngeal Airway Instructions For Use

WARNINGS

- Single use only
- The product should remain unopened until the point of use
- Sterile if package is unopened, undamaged and within shelf life
- Do not re-sterilise
- Do not expose to temperatures above 49°C
- This product must be in a pre-use condition and checked prior to use
- Do not use if the patient is suffering from a blocked nose, nasal congestion or nasal polyps
- Do not use if the patient has a history of epistaxis or fractured nasal bone

DIRECTIONS FOR USE

These directions are general guidelines intended for use only by qualified medical personnel. Any instructions and contraindications given are not exhaustive and it is the clinician's responsibility to ensure the safe, correct use of this product

- Select correct size of airway
- Lubricate outer airway tube with a suitable water based lubricant, ensuring that the distal tip remains patent
- Insert airway into patient's nasal cavity and ensure that the airway is operating correctly
- If oxygen is required, connect the oxygen tube onto the oxygen elbow before connecting the elbow to the airway
- To ensure a secure connection, use a push and twist action on all fittings.
- If the respiratory indicator is required, connect the hydrophobic filter / orange connector to a suitable male sampling line before connecting to an appropriate monitor
- When observing the respiratory rate, oxygen should only be administered at a flow rate of less than 8 LPM
- Any blocked or occluded devices should be removed immediately

STORAGE

Store product inside containers or outer boxes in a clean, dry area. Storage should be within a temperature of 10-30°C.

Do not expose to direct sun light or UV light

PRE-USE CHECKS

Do not use this product unless these checks are fully satisfactory.

- Check product is within its specified sterile shelf life
- Visually check the whole device for completeness, discolouration, damage or flaws
- Check that there is no blockage or occlusion in the airway

CLEANING INSTRUCTIONS

Device is single use only.
Do not reprocess.

naso-flo®

TECHNICAL INFORMATION AND RE-ORDERING CODES



0123

| Naso-Flo® with O ₂ Port, Soft Tip and 15mm Connector | | | Box of 10 |
|---|-----------------|------------------|-----------|
| Part Number | Inside Diameter | Outside Diameter | Length |
| NF104 | 4.0mm | 6.0mm | 110mm |
| NF105 | 5.0mm | 7.3mm | 125mm |
| NF106 | 6.0mm | 8.3mm | 145mm |
| NF107 | 7.0mm | 9.3mm | 155mm |
| NF108 | 8.0mm | 10.7mm | 185mm |
| NF109 | 9.0mm | 12.0mm | 205mm |

| Naso-Flo® with O ₂ Port, Respiratory Indicator, Filter, Soft Tip and 15mm Connector | | | Box of 10 |
|--|-----------------|------------------|-----------|
| Part Number | Inside Diameter | Outside Diameter | Length |
| NF204 | 4.0mm | 6.0mm | 110mm |
| NF205 | 5.0mm | 7.3mm | 125mm |
| NF206 | 6.0mm | 8.3mm | 145mm |
| NF207 | 7.0mm | 9.3mm | 155mm |
| NF208 | 8.0mm | 10.7mm | 185mm |
| NF209 | 9.0mm | 12.0mm | 205mm |

| Naso-Flo® with O ₂ Port, Respiratory Indicator - without Filter, Soft Tip and 15mm Connector | | | Box of 10 |
|---|-----------------|------------------|-----------|
| Part Number | Inside Diameter | Outside Diameter | Length |
| NF304 | 4.0mm | 6.0mm | 110mm |
| NF305 | 5.0mm | 7.3mm | 125mm |
| NF306 | 6.0mm | 8.3mm | 145mm |
| NF307 | 7.0mm | 9.3mm | 155mm |
| NF308 | 8.0mm | 10.7mm | 185mm |
| NF309 | 9.0mm | 12.0mm | 205mm |

| Naso-Flo® with O ₂ Port, Respiratory Indicator, Filter, Orange connector, Soft Tip and 15mm Connector | | | Box of 10 |
|--|-----------------|------------------|-----------|
| Part Number | Inside Diameter | Outside Diameter | Length |
| NF404 | 4.0mm | 6.0mm | 110mm |
| NF405 | 5.0mm | 7.3mm | 125mm |
| NF406 | 6.0mm | 8.3mm | 145mm |
| NF407 | 7.0mm | 9.3mm | 155mm |
| NF408 | 8.0mm | 10.7mm | 185mm |
| NF409 | 9.0mm | 12.0mm | 205mm |

| Naso-Flo® with O ₂ Port, Adjustable, Soft Tip and 15mm Connector | | | Box of 10 |
|---|-----------------|------------------|-----------|
| Part Number | Inside Diameter | Outside Diameter | Length |
| NF114 | 4.0mm | 6.0mm | 130mm |
| NF115 | 5.0mm | 7.3mm | 160mm |
| NF116 | 6.0mm | 8.3mm | 160mm |
| NF117 | 7.0mm | 9.3mm | 175mm |

| Naso-Flo® with O ₂ Port, Respiratory Indicator, Filter, Adjustable, Soft Tip and 15mm Connector | | | Box of 10 |
|--|-----------------|------------------|-----------|
| Part Number | Inside Diameter | Outside Diameter | Length |
| NF214 | 4.0mm | 6.0mm | 130mm |
| NF215 | 5.0mm | 7.3mm | 160mm |
| NF216 | 6.0mm | 8.3mm | 160mm |
| NF217 | 7.0mm | 9.3mm | 175mm |

| Naso-Flo® with O ₂ Port and Respiratory Indicator - without Filter, Adjustable, Soft Tip and 15mm Connector | | | Box of 10 |
|--|-----------------|------------------|-----------|
| Part Number | Inside Diameter | Outside Diameter | Length |
| NF314 | 4.0mm | 6.0mm | 130mm |
| NF315 | 5.0mm | 7.3mm | 160mm |
| NF316 | 6.0mm | 8.3mm | 160mm |
| NF317 | 7.0mm | 9.3mm | 175mm |

| Naso-Flo® with O ₂ Port, Respiratory Indicator, Filter, Orange connector, Adjustable, Soft Tip and 15mm Connector | | | Box of 10 |
|--|-----------------|------------------|-----------|
| Part Number | Inside Diameter | Outside Diameter | Length |
| NF414 | 4.0mm | 6.0mm | 130mm |
| NF415 | 5.0mm | 7.3mm | 160mm |
| NF416 | 6.0mm | 8.3mm | 160mm |
| NF417 | 7.0mm | 9.3mm | 175mm |



Tianjin Medis Medical Device Co., Ltd

No.15-A, Saida One Avenue,
Xiqing Economic Development Area, 300385 Tianjin, P.R.China
Tel: +86-22-83963862 Fax: +86-22-83988486
www: MEDIS-MEDICAL.com



Shanghai International Holding Corp. GmbH (Europe)
Eiffestrasse 80, 20537 Hamburg, Germany
Tel: +49-40-251317 Fax: +49-40-255726