

URGENT MEDICAL DEVICE RECALL NOTIFICATION

Thursday, August 27, 2015

Dear Valued Life-Assist Customer,

According to our records you have purchased an item that has been recalled by the manufacturer. Please examine your stock to determine if you have the following product with the affected lot number(s) in your possession.

Teleflex Medical Rüsç Flexi-slip™ Endotracheal Tube Stylet with Soft Distal Tip, 6FR

Life-Assist, Inc. Product Code	NDC/UPC	Lot #(s)		
AC900-06A	502501	12EE20	12KE48	14JG05
		12FE26	13BG26	14JE43
		12GE27	13BG36	14KG27
		12IE37	13DG24	14LG06
		12GE30	14FG03	15BG05
		13AG21	14GG01	15AG34
		12LG29	14GG03	15AG28
		12IE36	14HE32	15CG24
		12IE39	14HE34	15DE18
		13DG06	14IE36	15DG38
		12JE41	14KG11	15EG08

Reason for Recall: This recall has been initiated due to reports of the plastic coating of the stylet splitting and/or breaking off of the stylet. This may result in a piece of plastic totally or partially occluding the patient's airway and impairing ventilation, or necessitating invasive removal procedures in order to prevent complications such as atelectasis or pneumonia. No patient injuries have been reported related to this issue.

Teleflex Medical is recalling the affected lots in an effort to provide customers and their patients with the highest quality product possible.



We are now notifying our customers to take the following actions:

1. Immediately discontinue use and quarantine any products with the catalog numbers and lot numbers listed in this notification.
2. If you have affected stock, please contact Life-Assist Customer Service at 800-824-6016 or saleservice@life-assist.com to obtain a Return Authorization.
3. Complete the enclosed Recall Acknowledgement Form and return it, with any affected product, directly to:

Life-Assist, Inc.
ATTENTION: RECALL
11277 Sunrise Park Drive
Rancho Cordova, CA 95742

As of the date of this letter, Teleflex does not have any replacement product available to ship. Life-Assist is working closely with the Teleflex Recall Coordinator and Product Manager to get more information. Please contact Life-Assist Customer Service for updates.

We apologize for any inconvenience.

