

Exclusively Distributed By:  
**Mercury Medical**<sup>®</sup>

Your Need ... Our Innovation<sup>®</sup>

# ONLY ONE

*Innovative Airway Management Solution*

## The AnapnoGuard CUFFILL Accurate Cuff Pressure Control Made Easy

**The compact and intuitive AG CUFFILL device is an accurate solution for measuring both pressure and volume of airway cuffs in clinical settings.** Its pocket-size, syringe-like design allows a simple and easy operation by medical professionals, including first-responders and hospital staff, and by users in the home care environment.

The AG CUFFILL is designed to enhance patient safety by minimizing the potential of human error while measuring and inflating airway cuffs.



11300 - 49th Street North • Clearwater, Florida 33762-4807 USA

Tel. (800) 835.6633 • (727) 573.0088 • Fax (800) 990.6375

[www.mercurymed.com](http://www.mercurymed.com)

Since 1963

over ▶

# ONLY ONE

*Innovative Airway Management Solution*

## AG CUFFILL Highlights

- 10 cc volume
- Measurement range 0-99 (cmH<sub>2</sub>O)
- Measurement accuracy  $\pm 2$  (cmH<sub>2</sub>O)
- Disposable, enables 100 measurements
- FDA and CE approved



### ORDERING INFORMATION

Part #	Description	Packaged
#JRHSCUFF0041	AnapnoGuard Cuffill (AG Cuffill)	10/Box

Patent Pending

## Instructions for Use

- Turn the CUFFILL device ON by pressing the yellow button (it will stay ON for 60 seconds)

### For cuff pressure measurement

- The CUFFILL device should be connected to the tube when the plunger is pushed to the distal end of the syringe throughout the entire measurement

### For cuff inflation/deflation

- The CUFFILL device should be connected to the tube when the plunger is pulled back about 3 cc (or as required)
- The plunger should be pushed/pulled until the required cuff pressure is presented
- Cuff pressure will change with the breathing cycle
- Cuff pressure to be recorded is the lowest pressure presented



Manufactured By:



Exclusively Distributed By:

**MercuryMedical**<sup>®</sup>