# Laerdal Infant Airway Management Trainer

ENGLISH  
FRANÇAIS  
DEUTSCH  
ESPAÑOL  
ITALIANO  
NORSK  
SVENSKA  
SUOMI  
JAPANESE

Directions for Use  
Mode d'emploi  
Gebrauchsanweisung  
Instrucciones de utilización  
Istruzioni per l'Uso  
Bruksanvisning  
Bruksanvisning  
Käyttöohje

**Contents**

Cautions and warnings ..... 2  
 Introduction ..... 3  
 Instructions for use ..... 3  
 Maintenance ..... 4  
 Parts list ..... 6

*Product specifications are subject to change without notice.*

**CAUTIONS AND WARNINGS**

**Material compatibility**

Do not allow the manikin's skin to come in direct contact with ink or photocopied paper; as this can cause permanent stains on the skin.

Using protective gloves when handling the trainer will also reduce the risk of staining. However; avoid coloured vinyl or latex gloves as these may cause discolouration of the trainer's skin.

The face skin and airways must not be stored together with parts made in soft PVC, as this can damage the material used for the new parts.

**Upper airways**

Always make sure that the airways and tube have been properly lubricated before practising intubation on the trainer:

This is important both to provide the proper realism and to reduce the risk of damaging the manikin's upper airways.

## INTRODUCTION

The Laerdal Infant Airway Management Trainer is mounted on mounting base, and simulates a 3 month old infant head.

It is designed to provide the following common diagnosis and treatment modalities:

### Airway management

- Ventilation via bag valve mask
- Endotracheal and nasotracheal intubation
- Bilateral lung movement and stomach distention
- Oral/Nasal Airways
- Insertion of LMA (Laryngeal Mask Airway)

### Getting started

Before use, perform the following preparations:

### Lubrication of the airway

During this procedure, wear protective gloves to avoid contamination.

Spray into the mouth/airway using the lubricant spray (supplied with trainer). Also lubricate the tip of the endotracheal tube and laryngoscope blade before performing endotracheal or nasotracheal intubation.

## INSTRUCTIONS FOR USE

### Ventilation

### Airway open/closed - Stomach distension

In the flexed position, with the head resting forward onto the chest, the airways are closed. If an attempt is made to ventilate the manikin via bag mask between this position and the neutral position, air will not enter the lungs. If an air pressure of more than 10 mm Hg is applied, stomach distension can be observed on the trainer.

### Head tilt

In the neutral position, the airways are open. If ventilated via bag mask in this position, air will enter the lungs. If a too rapid ventilation is performed, causing an air pressure in excess of 10 mm Hg stomach distension will occur.

Hyperextension of the head/neck will not cause the airways to close.

### Jaw thrust

The trainer also allows jaw thrust to be performed. Performed in the flexed position, the airways will open and allow ventilation by bag mask.

### Insertion of LMA

Insertion of LMA can be practiced. A size 1 LMA is recommended.

### Intubation

The trainer can be intubated via the oral and nasal routes.

A 3,5 mm tube is recommended to avoid backward air leakage. A No. 1 laryngoscope blade is recommended.

If intubated too deeply, the tube will pass into the right mainstem bronchus, causing ventilation to the right lung only.

### Important:

*Always make sure that the airways and tube have been properly lubricated before practising intubation on the trainer.*

*This is important both to provide the proper realism and to reduce the risk of damaging the upper airways. Use the lubricant spray supplied with Infant Airway Management Trainer.*

The cricoid pressure technique (Sellick manoeuvre) can be realistically performed on the Infant Airway Management Trainer:

### Lungs, stomach

The trainer contains two separate lungs and a stomach.

They are designed to provide movement during ventilation of either lung or distention of the stomach.

To expel air from the stomach, simply press on the

stomach part.

The trainer will expel all air back through the mouth and nose.

### Airway hygiene

Please note that if mouth to mouth or mouth to nose ventilation has been performed, a thorough cleaning of the upper airways, and a replacement of the lungs and stomach are necessary.

See section "Cleaning upper airways".

However, we recommend that this trainer not be used for mouth to mouth or mouth to nose ventilation.

## MAINTENANCE

### Lubrication

It is essential that the upper airways are properly lubricated in order to obtain the intended realism and durability.

Take care to lubricate the airways, endotracheal tube and laryngoscope blade as described in section "Lubrication of the airways" on page 3 before using the manikin.

### Replacing or cleaning the face skin

To remove

1. Detach face from retainer at each ear.
2. Lift off face.

To clean

Clean face skin as follows:

Immerse face skin into a disinfectant solution\*.

Scrub stubborn stains when necessary and leave for 10 minutes.

Rinse with fresh water:

Allow face skin to dry.

\*Note:

*Virkon can be used.*

*Depending on local policies you may prefer to use a freshly prepared sodium hypochlorite solution. Local cleaning and handling procedures must then be followed.*

Alternatively replace with new face skin (Cat.No.08 33 10).

To install

1. Fold face upward to avoid direct finger contact with mouth and nose area.

2. Unfold face over head.

3. Fasten retainer at each ear.

### Replacing the lungs/stomach

To remove

Remove stomach pressure valve band. Unsnap and discard the lungs and stomach.


To install

Unpack new lung/stomach (Cat. No. 250 20250).


Connect stomach to oesophagus end of head/airway assembly.

With the stomach positioned outside the "body", fold the lungs back into the body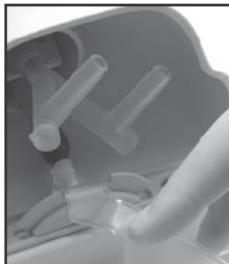

Connect the two lungs to the carina (Y piece).

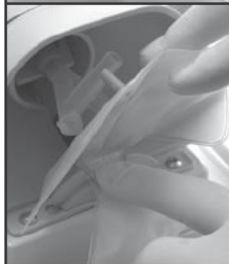Mount the stomach pressure valve band over the two tabs.

### Cleaning upper airways

To remove

Press down tab on inner neck connector and pull head away from body part. Remove face skin as described in "To remove" in section "Replacing or cleaning the face skin".

Unsnap and discard the lungs and stomach.

To clean

Open and empty the cleaning kit basin (optional Cat. No. 081500).

Place the trainer head face down in the cleaning basin.

Connect the two lung tube connectors to the double Y piece in the tube system.

Connect the oesophagus to the single connection in the tube system.

Insert the cleaning syringe into the valve opening and place the free end of the tube system into the basin.

Fill the basin with liquid to just underneath the edge. Circulate liquid through the manikin airways by pumping the syringe plunger.

Use this setup to go through the following steps:

1. Use a Disinfectant solution\*. Airways should remain completely filled for at least 10 minutes.
2. Use fresh water to remove disinfectant solution.

After each step lift the head clear of the liquid to allow drainage.

3. Allow to dry completely before reconnecting head to body part.

\* Virkan can be used.

*Depending on local procedures you may prefer to use a sodium hypochlorite solution freshly prepared for single class use. This solution should have at least 500 ppm free available chlorine, i.e. 22 ml of 2.5 % household bleach per 500 ml of water. Local cleaning and handling procedures must then be followed.*

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