


The original USA made

Little Sucker[®] Nasal Tip

Nasal Suction Device

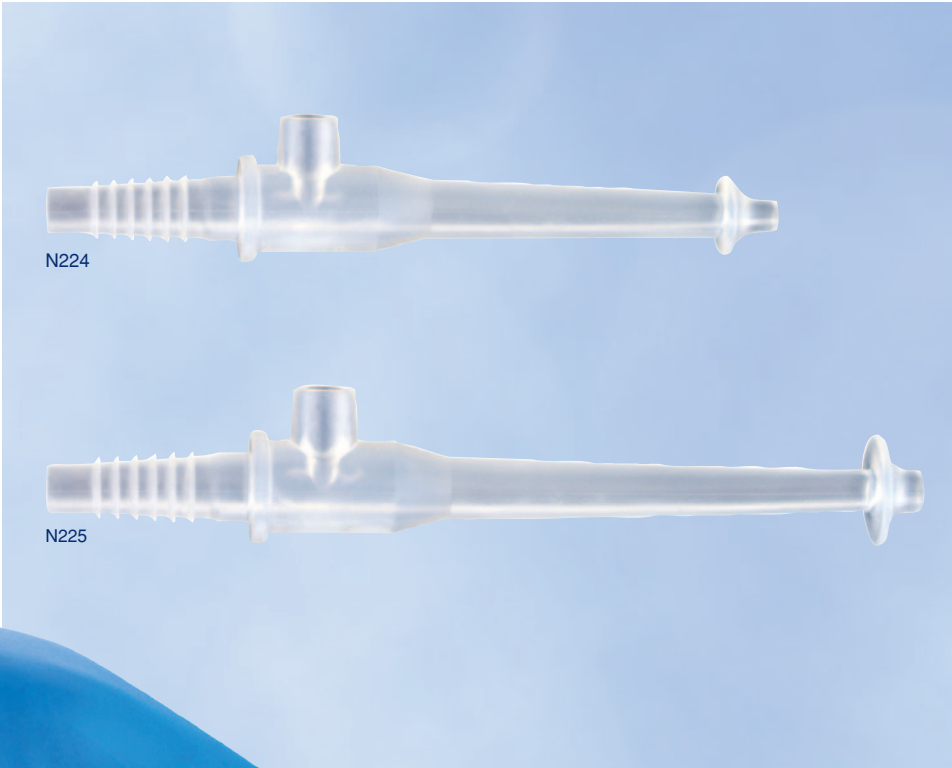

FEATURES AND BENEFITS

- Specialized tip designed to minimize potential tissue trauma
- Designed for single handed suctioning
- Thumb port for intermittent suctioning
- Soft, flexible tip similar to a bulb syringe
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

Cat No.	Item	Qty/Unit
N224	Preemie Nasal Tip	50/box
N224TP	Preemie 2-piece	50/box
N225	Standard Nasal Tip	50/box
N225TP	Standard 2-piece	50/box



U.S. Patent
 #7,635,361; # D668,757;
 #6,958,050; #D474,269

Little Sucker® Nasal Tip

Nasal Suction Device

INDICATIONS FOR USE

The Little Sucker Nasal Tip is indicated for use as a vacuum powered suction device to aspirate fluids from the nares.

DIRECTIONS FOR USE

Step 1

Connect barbed end of Little Sucker Nasal Tip to suction line.

Step 2

Set suction to appropriate pressure per hospital protocol or physician recommendation.

Step 3

Block thumb port with thumb to begin suctioning.

Step 4

Suction nares with intermittent suctioning.

Note: For intermittent suctioning, unblock thumb port to stop suctioning. Block thumb port again to continue suctioning.

Step 5

Suction sterile water through Little Sucker Nasal Tip after each use.

Step 6

Replace Little Sucker Nasal Tip every 24 hours or per hospital protocol, whichever is sooner.

GENERAL INFORMATION

Product Materials

Little Sucker: TPE (thermoplastic elastomer)
Made with natural rubber latex: No
Made with plasticizer DEHP: No

Size

N224 Preemie
Length: 106 mm
Tip opening diameter: 2 mm
Nasal guard diameter: 11 mm
Thumb port opening diameter: 5 mm

N224TP Preemie Length: 109 mm
Tip opening diameter: 2 mm
Nasal guard diameter: 11 mm
Thumb port opening diameter: 6 mm

N225 Standard
Length: 120 mm
Tip opening diameter: 3 mm
Nasal guard diameter: 15 mm
Thumb port opening diameter: 5 mm

N225TP Standard
Length: 109 mm
Tip opening diameter: 2 mm
Nasal guard diameter: 15 mm
Thumb port opening diameter: 6 mm

Packaging

Individually packaged: Yes
Package type: Polyethylene bag
Dispenser box: Yes

Visit neotechproducts.com for the most up-to-date information.

Polyethylene bags are biodegradable. Boxes are printed with environmentally friendly soy ink on recyclable material.

CAUTIONS & WARNINGS

Federal law restricts this device to sale by or on the order of a physician.

For nasal use only.

Inserting the Little Sucker Nasal Tip too far into the nares can cause tissue trauma.

The Little Sucker Nasal Tip should remain free of debris.

Single patient use only.



For more information or to order your FREE sample call
1-800-966-0500 or visit neotechproducts.com

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355