

# 8000AA /AP

## SpO<sub>2</sub> Clip Sensors

Use **ONLY** with compatible Nonin Medical pulse oximeter monitors.

### 1. Select the proper size sensor.\*

Adult **8000AA** (blue): 0.5 – 1 in (12.5 – 25.5 mm)


Pediatric **8000AP** (green): 0.3 – 0.5 in  
(7.5 – 12.7 mm)

**2. Use clean, dry finger where sensor will be placed.** Heavy lotions, sunscreen, dirt, oils and nail polish may alter accuracy.

**3. Gently squeeze to open the sensor, and insert the finger all the way into the sensor.** The finger symbol should be on the top of the finger. Check that sensor gently hugs finger, and is not too tight or too loose.

**4. Secure sensor to back of hand with medical tape to allow movement if necessary.**

**5. Connect to a Nonin pulse oximeter.** Follow Operator's Manual for oximeter to measure oxygen saturation and pulse rate.

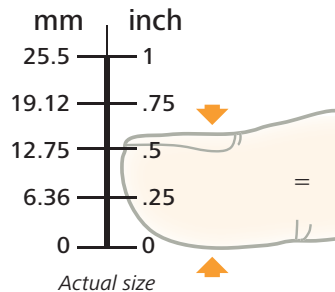

\* Sizing is for reference only. Fit, patient comfort and signal quality should be used to determine appropriate size/model.

- A. Use index, middle or ring finger. Rotate finger sites for extended use and comfort as needed.
- B. Do not use on arm with IV line or BP cuff.
- C. Check sensor position and skin at site every 6 to 8 hours.
- D. If signal quality is low on the oximeter:
  - 1. Check sensor position and size for good fit
  - 2. Check connection at oximeter
  - 3. Check hand is warm, and has good blood flow
  - 4. Use a different finger
- E. Clean sensor between patients or as needed.
  - 1. Use soft cloth with 10% bleach/90% water solution (household bleach < 10% sodium hypochlorite)
  - 2. Do not use ammonium chloride based cleaning fluids.
  - 3. Open sensor up to 90 degrees to clean skin contacting surfaces. Do not overextend. The spring on the AP model can be released for easier access.
  - 4. Wipe away from connector plug. Do not submerge or immerse sensor or connector plug.
  - 5. Allow to completely dry before next use.

**A short-cable model of the 8000AA Sensor is available for use with the WristOx<sub>2</sub>® Model 3150 Pulse Oximeter.**

THIS GUIDE IS FOR REFERENCE ONLY. Refer to the Instructions for Use for complete information, warnings, cautions and specifications for the product you are using.

Visit [nonin.com](http://nonin.com) for more information and video tutorials.



[nonin.com/sensors](http://nonin.com/sensors)

**Nonin Medical, Inc.**  
13700 1st Avenue North  
Plymouth, MN 55441-5443 • USA  
**Tel:** +1.763.553.9968  
+1.800.356.8874  
**Fax:** +1.763.577.5521  
**E-mail:** [info@nonin.com](mailto:info@nonin.com)

**Nonin Medical B.V.**  
Prins Hendriklaan 26  
1075 BD Amsterdam • Netherlands  
**Tel:** +31 (0)13-79 99 040  
**Fax:** +31 (0)13-79 99 042  
**E-mail:** [infointl@nonin.com](mailto:infointl@nonin.com)